

Invacare® | **Slings**




Optimal positioning and ease of application

Invacare's new range of slings provides a safe and comfortable transfer solution for many types of physical disability. This selection of high quality slings has been developed with the collaboration of Occupational and Physiotherapists, to ensure optimal fit, support for the client and ease of application for the carer.

Most slings are available in five sizes with the option of fabric types. A range of adjustments enable the slings to meet the specific needs of the individual.



Yes, you can.®

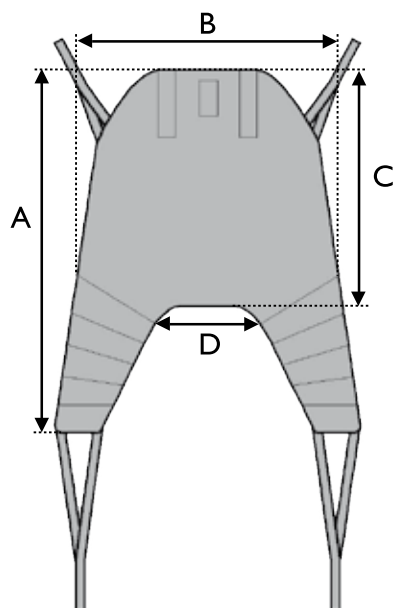


Invacare **Universal High Sling**

Supports the whole body and the head giving the client a comfortable and slightly reclined position.

The **Universal High Sling** is typically used for those who have limited control of the head, trunk or hips – or for transfers to and from a lying position. The **Universal High Sling** can also be used for seated to seated transfers. This sling can also be used without the head support, by simply removing the plastic rigidisers.

Size	A	B	C	D	Product number / Material		
					Polyester fabric	Net fabric	Spacer fabric
XS	1040	740	610	340	1483776	1485950	1485955
S	1080	840	730	380	1483777	1485951	1485956
M	1420	1000	830	400	1483778	1485952	1485957
L	1550	1100	920	510	1483779	1485953	1485958
XL	1680	1150	920	550	1485949	1485954	1485959

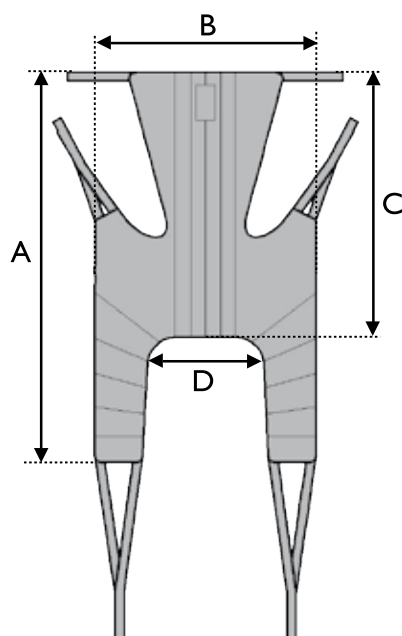


Invacare **Universal High Plus Sling**

Supports the whole body including the head giving the client a comfortable and slightly reclined position. The **Universal High Plus sling** also allows clients keep their arms outside the sling. It is typically used for those who have limited control of the head, trunk or hips – or for transfers to and from a lying position. The **Universal High sling** can also be used for seated to seated transfers. This model has a stiffened fabric in the back section, to ensure optimal weight distribution and comfort for the client.




Size	A	B	C	D	Product number / Material	
					Polyester fabric	
S	1380	840	880	340	1485960	
M	1510	950	930	450	1485961	
L	1580	1100	970	560	1485962	
XL	1590	1190	970	630	1485963	

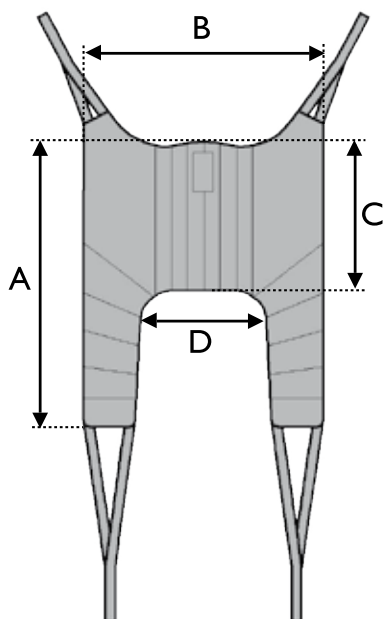

Invacare **Universal Standard Sling**

Supports the whole body; including the shoulders. This sling should only be used for those who have good head control. The **Universal Standard sling** is often preferred as an all-round sling for the elderly, as it supports the entire body and gives the client a feeling of comfort as well as an unobstructed view. This sling can be used for seated to seated transfers and is easy to apply. This model also has a stiffened fabric in the back section, to ensure optimal weight distribution and comfort for the client.


Size	A	B	C	D	Product number / Material		
					Polyester fabric	Net fabric	Spacer fabric
S	1000	920	510	410	1485882	1478411	1485363
M	1090	1050	560	480	1485883	1478412	1485364
L	1180	1090	580	560	1485884	1478413	1485365
XL	1240	1250	585	710	1485885	1478414	1485366

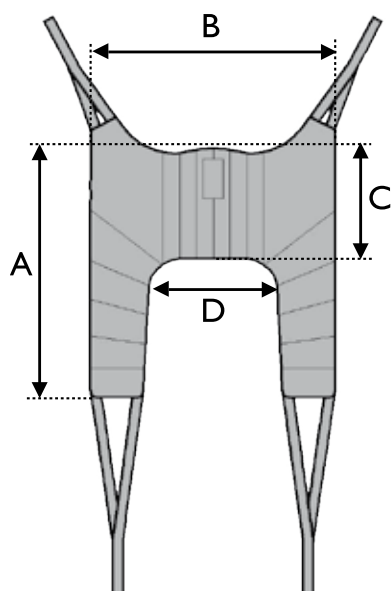

Invacare **Universal Low Sling**

Supports the hips and lower back. The **Universal Low sling** can be used for those who have some trunk control and full head control. It is used for seated to seated transfers and is easy to apply. This sling also has stiffened back section to ensure optimal weight distribution and comfort for the client.


Size	A	B	C	D	Product number / Material	
					Polyester fabric	
S	850	800	350	410	1491893	
M	920	940	375	480	1491894	
L	950	1075	405	575	1451067	

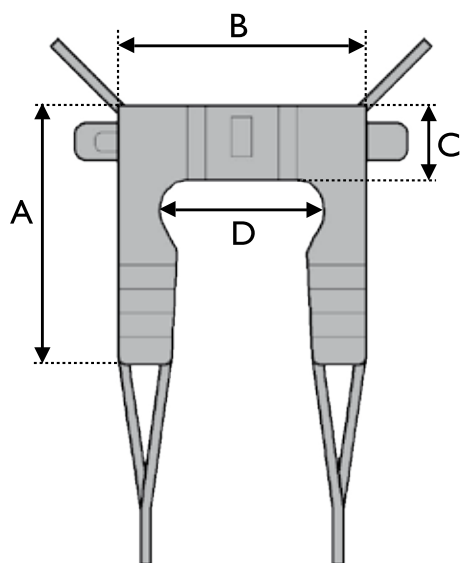

Invacare **Dress Toileting Sling**

Within the Invacare sling range there are two toilet transfer slings available; with or without head support. Both models come with a belt for increased support around the body and for safe positioning. The **Dress Toileting Sling** is suitable for those who have limited mobility. The model with head support is designed for clients who also have reduced head control. The **Dress Toileting Sling** is suitable for those who want to be able to dress and undress during the toilet transfer. It can be used for seated to seated transfers and is very easy to apply.

Size	A	B	C	D	Product number / Material	
					Toileting polyester fabric	Toileting High polyester fabric
S	870	710	200	430	1478440	1478444
M	950	800	230	490	1478441	1478445
L	1020	900	230	600	1478442	1478446
XL	1100	1080	270	760	1478443	1484780

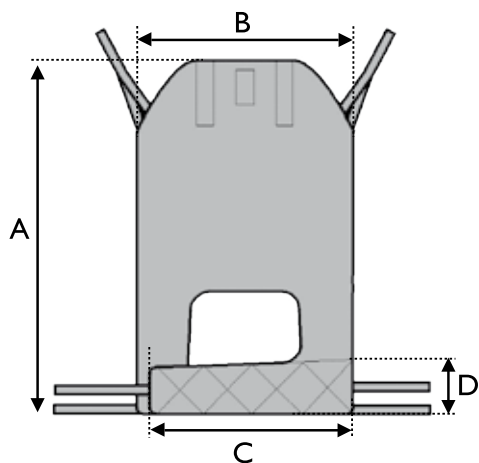


Invacare **Easy-fit Sling**

Supports the whole body and the head, and is therefore suited for clients with the most challenging physical disabilities*. The specially designed separate leg supports are easy to apply, and the sling affords a good hip position, which allows the client to sit relatively upright. The **Easy-fit sling** is also well-suited for amputees. It is easy to apply in both sitting and lying positions.


Size	A	B	C	D	Product number / Material	
					Polyester fabric	Spacer fabric
XS	1100	710	595	170	1451101	1451106
S	1130	760	640	180	1451102	1451107
M	1200	800	675	190	1451103	1451108
L	1270	860	720	210	1451104	1451109
XL	1390	940	780	220	1451105	1451110

* Following risk assessment

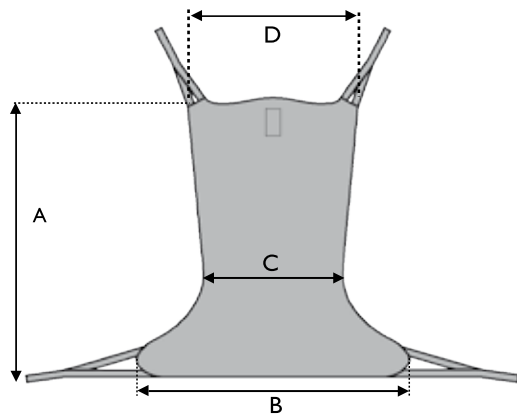


Invacare Comfort Sling, Comfort Sling with Toilet Opening and Comfort Standard Sling

This sling supports the whole body and the head. The **Invacare Comfort Sling** is ideally suited to those who have chronic pain. The sling is also useful for above knee amputees, or for those who due to postural position or pain prefer to remain in a seated position. The **Comfort Sling** is applied while the client is lying down, and is used to transfer the client into a seated position. The **Comfort Sling with Toilet Opening** is applied in the same way and is used for toileting.

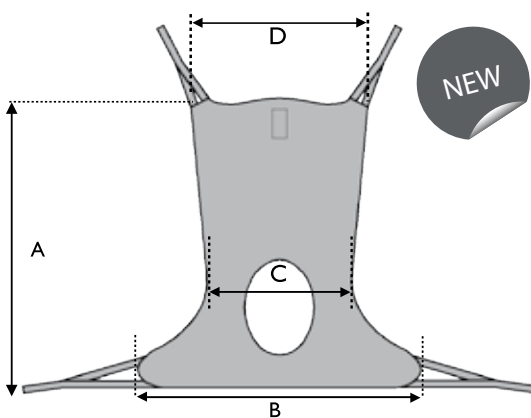

Invacare **Comfort Sling** with and without Toileting Opening


Size	A	B	C	D	Product number / Material			
					Comfort		Comfort Toileting	
					Net fabric	Spacer fabric	Polyester fabric	Net fabric
XS	1040	970	510	960	1485967	1485948	1451091	1451095
S	1150	1050	550	1020	1485968	1485554	1451092	1451097
M	1395	1170	610	1160	1485969	1485777	1451093	1451098
L	1445	1290	675	1280	1485970	1485778	1451094	1451099
XL	1460	1330	745	1470	1485970	1451090	1451095	1451100

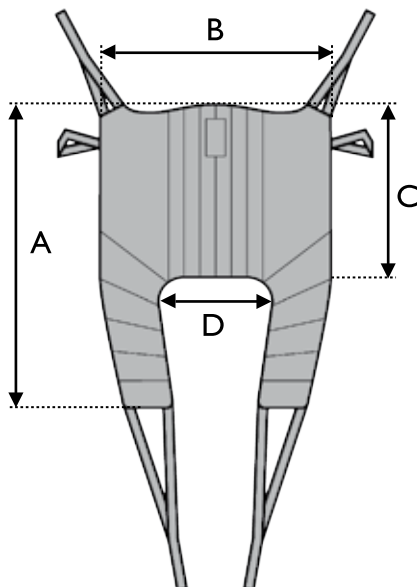

Invacare Amputee Sling

This model is specially designed for the safe transfer of clients with single or double amputations. The sling affords the client comfortable support in a slightly reclined position. It is important that the client has good head control.

Size	A	B	C	D	Product number / Material
					Polyester fabric
S	1100	930	490	320	1485367
M	1160	970	520	360	1485368
L	1360	1230	570	430	1485369
XL	1400	1350	570	480	1485370

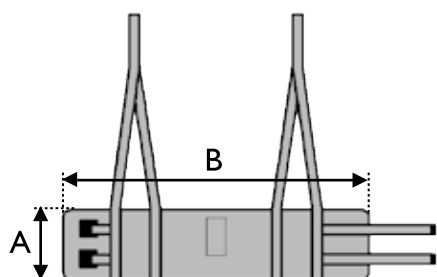


Invacare **Standing Transfer Vest**

The **Standing Transfer Vest** is applied around the trunk. The client's feet must be on the floor and the client must also have some standing ability. The **Standing Transfer Vest** is used for those with reduced balance and/or walking ability. The vest is especially useful for toileting. It also reduces the risk of injury to the carer.

A groin support is available as an accessory to the **Standing Transfer Vest**. This helps keep the vest in place.

Invacare **Standing Transfer Vest**

Size	A	B	Product number / Material	
			Polyester fabric	Groin Band
XS	150	750	1485404	–
S	150	850	1485405	1492662
M	180	950	1485406	–
L	180	1050	1484683	1492663
XL	180	1200	1484684	–

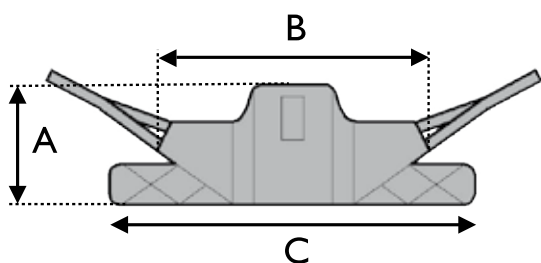


Invacare **Stand Assist Sling**

Designed for use with the **Reliant 350 lifter**, but compatible with other types of **Stand Assist lifter***, the **Invacare Stand Assist Sling** offers a comfortable and safe transfer for clients who have some weight bearing ability, but cannot stand independently. In addition, the client should be coherent, have good body and head control and the ability to grip the lift boom during the transfer. The sling is well padded and comes with non-slip material on the back to ensure a comfortable transfer position.

Size	A	B	C	Product number / Material	
				Polyester fabric	
S	260	840	950	1484685	
M	330	895	1170	1484686	
L	360	950	1270	1484687	
XL	360	1020	1810	1484688	

May be suitable for use with **Stand Assist Lifters** that are compatible with fabric loop slings. Following risk assessment.

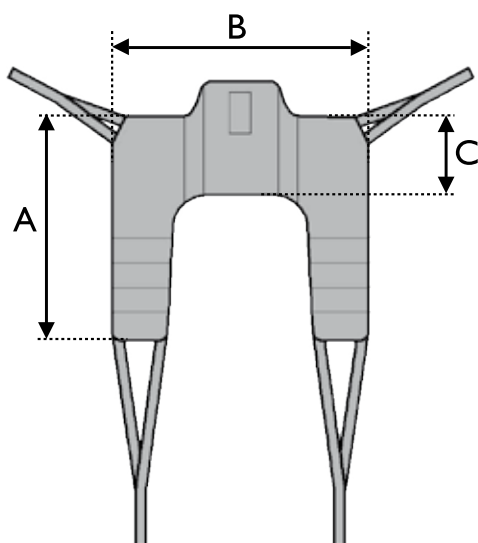


Invacare **Transfer Stand Assist Sling**

The *Invacare Transfer Stand Assist Sling* can be used with the **Reliant 350** lifter and enables a simple seated transfer. This sling is used for those with good body and head control who are able to grip the lift boom.

				Product number / Material
Size	A	B	C	Polyester fabric
S	860	840	240	1484689
M	1020	895	260	1484690
L	1180	950	330	1484691
XL	1230	1020	350	1485117


Sling Accessories

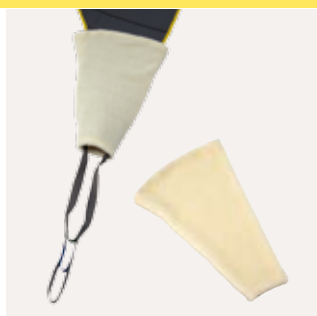

Fleece padding for extra comfort



Leg insert, Foam. For **Universal Standard, Universal High** and **High Plus** in Net Material

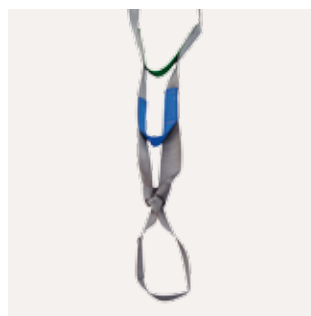

Extension loops - in order to give more positioning possibilities



Head support stiffeners - for Slings with head support. **Easy Fit, Universal High**

Sling Types	Fabric Matrix			Size Matrix				
	Solid	Net	Spacer	XS	S	M	L	XL
Universal Low	●				●	●	●	
Universal Standard	●	●	●		●	●	●	●
Universal High	●	●	●	●	●	●	●	●
Universal High Plus	●	● <small>NEW</small>	● <small>NEW</small>		●	●	●	● ¹
Dress Toileting	●				●	●	●	●
Dress Toileting High	●				●	●	●	●
Easy-fit	●		●	●	●	●	●	●
Amputee	●				●	●	●	●
Comfort		●	●	●	●	●	●	●
Comfort Toilet	●	●		●	●	●	●	●
Stand Assist	●				●	●	●	●
Transfer Stand Assist	●				●	●	●	●

1/ Only available in solid material

Fabrics



Solid Polyester

A standard material that is easy to apply – slides smoothly and dries quickly after washing.



Net*

Used for bathing or if the user will stay seated on the sling for a period of time.




Spacer*

A slightly elastic fabric that adapts to the user's body shape. Well suited for those with skin integrity issues.

* It is important to carry out a risk assessment with reference to skin integrity for those who be seated on a sling for long periods.

INVACARE® Limited
Pencoed Technology Park - Pencoed -
Bridgend CF35 5HZ
Tel: +44 1 656 776 222
Fax: +44 1656 776 220
E-mail: uk@invacare.com
www.invacare.co.uk
Sales Order E-mail:
ordersuk@invacare.com

Size guide

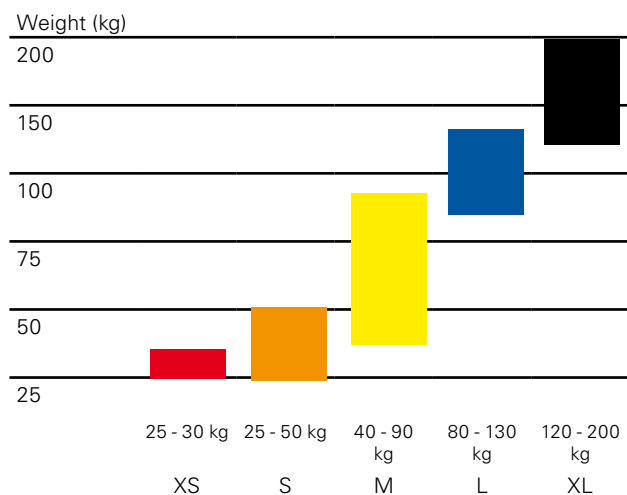

1 year warranty



200 kg**



95°



All slings should be tested and adjusted to the individual user to ensure optimal comfort and safety during transfers.

** Except for stand assist, Transfer stand assist & Standing transfer vest in combination with groin band: the safe working load is 159 kg.

INVACARE® Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road - SWORDS - County
Dublin
Tel. +353 1 8107084
Fax +353 1 8107085
Email: Ireland@invacare.com
www.invacare.ie

© Invacare® International Sarl 2010
All rights reserved.

All measurement have been done under the Invacare® standard test conditions, unless quoted differently. All information quoted is believed to be correct at time of print.
Invacare® reserves the right to alter product specifications without prior consultation.

Slings - UK - 10/2010 - 1503851

