

Sitter Harness Installation

VISUAL AID

List of Straps

Lay out the straps. You should find that you have 4 separate pieces: All parts are (sub-numbered) to help identify the proper orientation.

A: chest strap (A-1 & A-2)


B: crotch strap with loop (B-1 & B-2)

C: waist strap with a buckle (C-1 & C-2)


D: attachment strap (D-1 & D-2)




1. Chest Strap

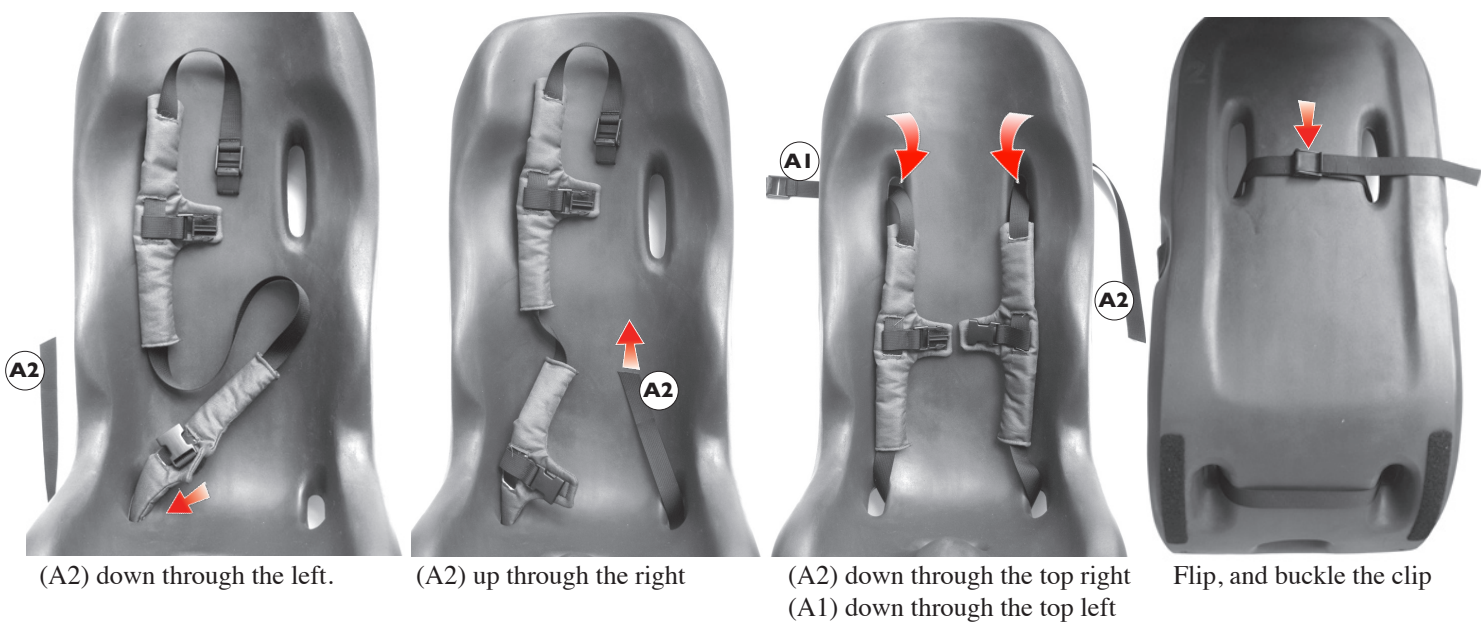


2. Crotch Loop & Waist strap

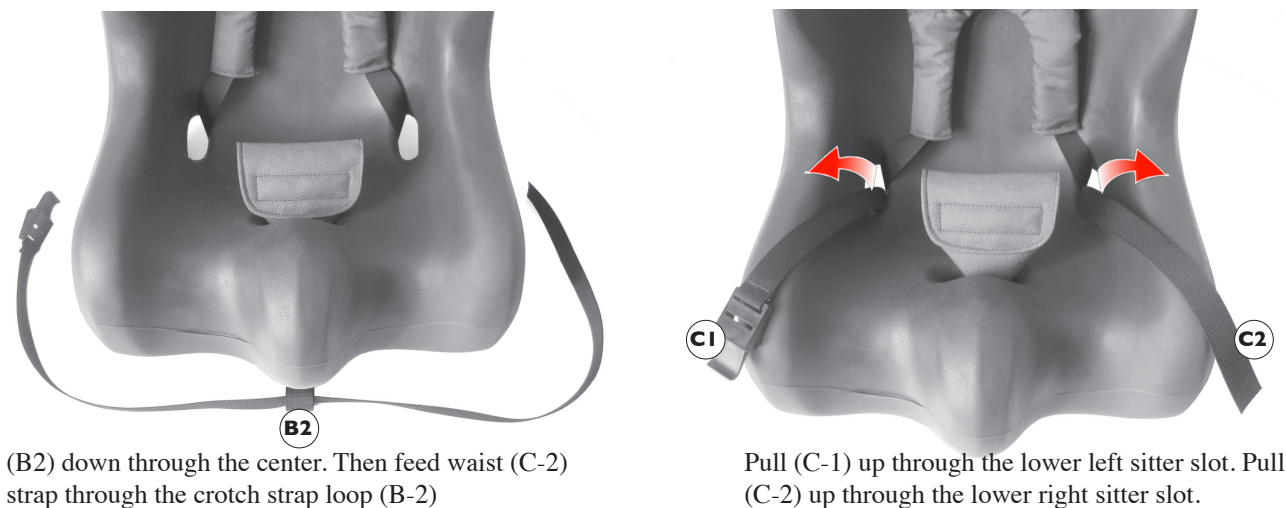

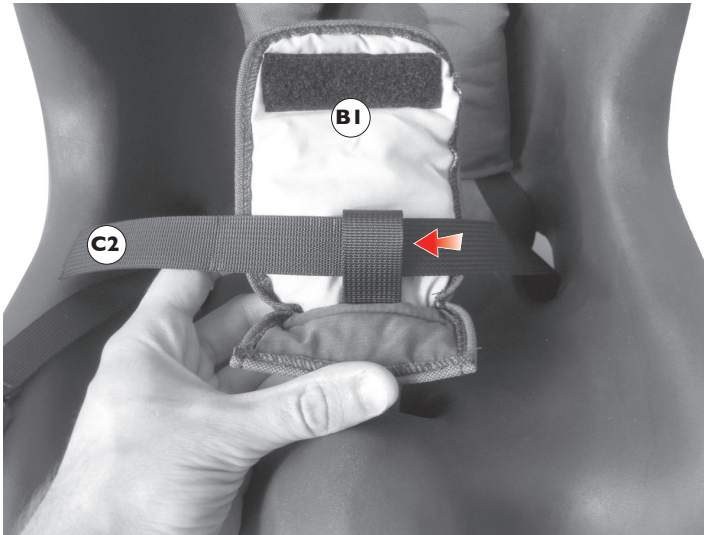


Open (B1) and thread (C-2) through crotch loop.

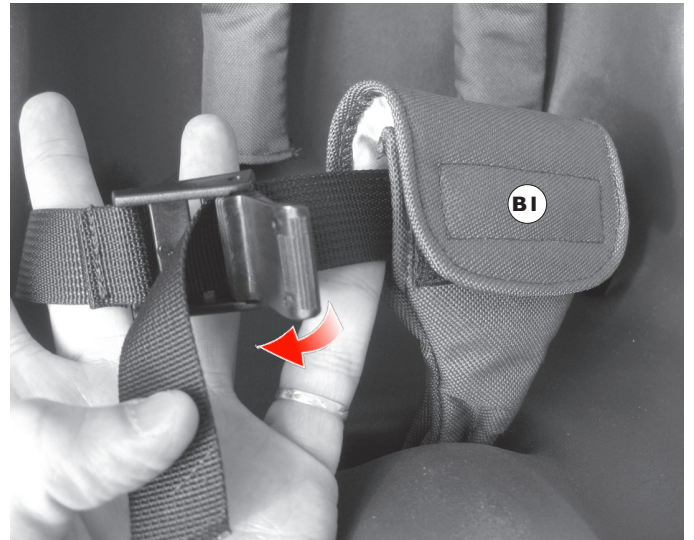


Close (B1). Buckle waist strap by connecting (C-1) & (C-2). Close

3. Attachment Strap

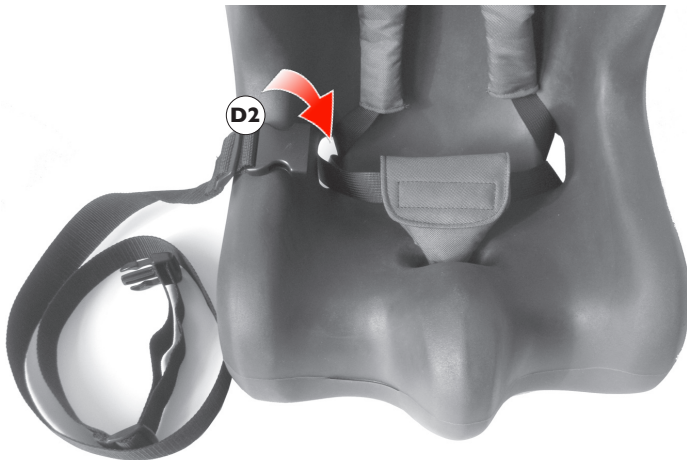

Thread (D-2) down through the lower left sitter slot.

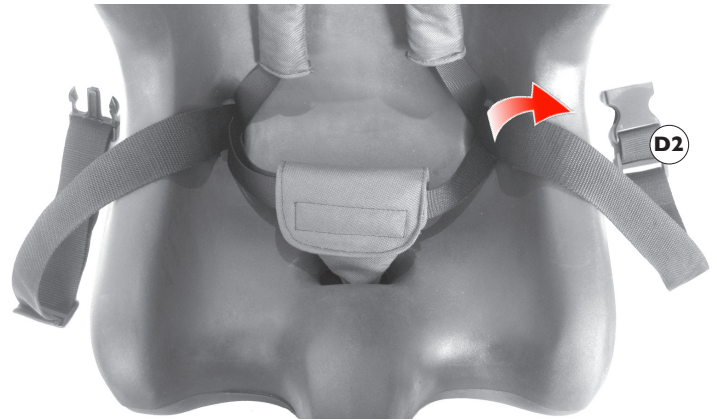

Thread (D-2) up through the lower right sitter slot.



Bergeron By Design
15 South Second Street
Dolgeville, New York 13329
T | 315 429 8407
F | 315 429 8862



© 2009 Bergeron By Design
The information contained herein is subject to change without notice.
Bergeron By Design shall not be liable for technical errors or omissions herein.