



# Etac Manual Transfer System

International

 **etac**<sup>®</sup>

Creating Possibilities





## Welcome to Etac!

Etac takes great pride in being a company that develops products that are among the best to be found in regards of design, function and quality. We provide a wide range of institutional and home health care products and market them globally. All Etac's products are developed in close co-operation with industrial designers, care providers, therapists and end users. We supply products in the following categories: Hygiene, Manual Wheelchairs, Power Wheelchairs, Patient Lifts, Manual transfer and Small Aids for Daily Living. Products that help people with their daily living routines regardless of their physical circumstances.

Please visit us at [www.etac.com/exp!](http://www.etac.com/exp!)



## History of Etac

The humanist basic idea behind our founder RFSU lives on with us at Etac also today.

The story of Etac begins in Sweden, where there was a prohibition against selling birth control without special permission. In order to make contraceptives more readily available therefore started RFSU so-called medical stores. To broaden the range is supplemented by including assistive technology.

This sowed the seeds to start their own development tools. 1972 began to manufacture quality products, with carefully-considered form and function. Thus began a new era in our industry.

Etac is owned by Nordstjernan.



## Winning design for three decades

Our products have been developed in close cooperation with established designers and rehab staff and users. This has resulted in an ergonomic design and quality that is unique in the market. As evidence of the products good shape, we have over the years received a number of Swedish and international design awards.



red**dot**



good design award



Excellent Swedish Design



# Content

## Support

Etac SupportBelt	7
Etac Sling	8
Etac SmartMove	9
Etac BedString	10
Etac HandyGrip	11

## Transfer – Sitting

Etac Turner	13
Etac Transfer Boards	14-15
Etac PediTurn	16
Etac Glide Pads	17

## Transfer – Horizontal

Etac 2Move	19
Etac Transfer Mattress	20

## Positioning – Sitting

Etac GlideCushion	23
Etac Swivel Cushions	24

## Positioning – Horizontal

Etac MultiGlide Glove	27
Etac MultiGlide	28-29
Etac MultiGlide with handles	30
Etac GlideCushion	31
Etac TurningMattress	31
Etac SatinSheet System	32-33
Etac In2Sheet	34
Etac 4WayGlide Mini & Midi	35
Etac 4WayGlide LPL & LPL Maxi	36-37
Etac TwinSheet	38
Etac TwinSheet 4Glide	39
Etac AirGlide LPL	40
Etac OneManSling	42
Etac Positioning Wedge	43

## Special Solutions

Etac SlingOn	45
Etac WiseMover	46
Etac TransSeat	47
Etac MiniBoard	48
Etac X-ray	50-51
Etac Rescue	52
Etac HipTurner	54
Etac Non-slip	55
Etac CloverFoot	56



# Support

Tools to help and support in mobilization and transfer.

# Etac SupportBelt

## Support belt with handles

Etac SupportBelt is used to offer support and assistance during mobilization and transfers. The diagonally placed handles allow the caregiver to maintain a more ergonomic grip.


SupportBelt is a useful tool when helping a person to stand up or sit down, to support walking or to assist during transfer between bed, wheelchair, chair, etc. SupportBelt has non-slip material on the inside and an adjustable belt buckle.

*Etac SupportBelt has diagonally placed handles that allows the caregiver to maintain a more ergonomic grip.*



### Intended Use

- Support stand up / sit down
- Support walking training
- Support transfer between bed, wheelchair, chair, etc
- Transfer from wheelchair to toilet

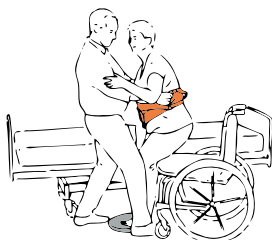

### Description

	Width x Length		Art.no
	Cm	Inch	
SupportBelt XS, 4 handles, waist size 65-105 cm	9 x 105	3.5 x 41.3	IM8010509
SupportBelt S, 5 handles, waist size 65-105 cm	13 x 105	5.1 x 41.3	IM8010513
SupportBelt M, 6 handles, waist size 90-130 cm	13 x 130	5.1 x 51.2	IM8013013
SupportBelt L, 8 handles, waist size 125-150 cm	13 x 150	5.1 x 59.1	IM8015013
SupportBelt XL, 11 handles, waist size 145-170 cm	15 x 170	5.9 x 66.9	IM8017015

### Specifications

Outside: Nylon  
 Inside: Nylon with PU-coating  
 Padding: Polyetylen  
 Handles: Polyester strap with polypropylen  
 Belt Buckle: Acetal, nylon



### Washing Instructions

Can be disinfected with LT-CIN isopropylalcohol 72 %).  
 Do not use fabric softener or bleach.



### Colourcode - size labeling

	XS
	S
	M
	L
	XL



# Etac Sling

## Sling with handles

Etac Sling with handles acts as extension of the arms, allowing the caregiver to offer support and assistance whilst maintaining a good ergonomic position and grip, without encroaching on the service user's personal space.



Etac Sling is a flexible tool that allows the caregiver to maintain a more ergonomic grip. It can be used to support many different transfers such as, moving higher up in bed, transfer between sitting positions, etc. Sling has 4 handles on each side. The inside has a non-slip material to help prevent sliding out of position during the transfer, whilst the outside has a low friction surface to aid placement and removal



*Etac Sling is a flexible tool offering a number of different functions.*

## Intended Use

- Support moving higher up in bed
- Support transfer between bed, wheelchair, chair, etc
- Support stand up / sit down
- Moving back / forward in chair / wheelchair

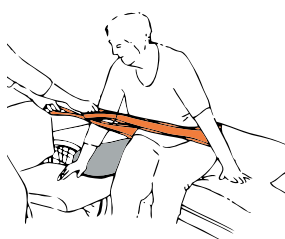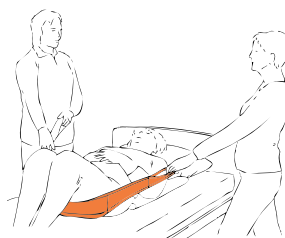

### Description

Sling Long, 2 x 4 handles  
Sling Short, 2 x 4 handles

### Width x Length

Cm	Inch
16 x 190	6.3 x 74.8
16 x 120	6.3 x 47.2

### Art.no

IM428  
IM428K

### Specifications

Backside: Nylon  
Padding: Polyester  
Inside: Nylon with polyurethan  
Handles: Polyester



### Washing Instructions

Do not use fabric softener or bleach.



# Etac SmartMove

## Support pad with handles

Etac SmartMove acts as extension of the arms, allowing the caregiver to offer support and assistance whilst maintaining a good ergonomic position and grip.


Etac SmartMove pad can be used to support and help transfer from sitting to standing, standing to sitting, to move higher up in bed or move back / forward in a chair. SmartMove provides the caregiver with a place to grip and to maintain a good working position. The inside has a non-slip material to help prevent sliding out of position during the transfer, whilst the outside has a low friction side to aid placement and removal.



*Etac SmartMove has a non-slip material to help prevent sliding out of position.*

## Intended Use

- Support stand up / sit down
- Support moving higher up in bed
- Moving back / forward in chair / wheelchair
- Support transfer between bed, wheelchair, chair, etc

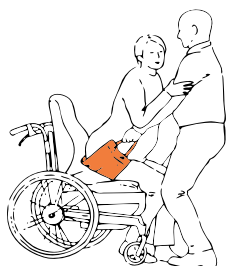

Description	Width x Length		Art.no
	Cm	Inch	
SmartMove 2Handles	21 x 60	8.3 x 23.6	IM425
SmartMove 4Handles	21 x 60	8.3 x 23.6	IM426

### Specifications

Outside: Nylon with polyurethancoating

Inside: Polyester

Handles: Polyester

Underside: Polyester



### Washing Instructions

Do not use fabric softener or bleach.



# Etac BedString

## String with handles

Etac BedString with handles is mainly intended for the active user to support moving to a sitting position in bed. In the sitting position, the user may use it to lean themselves forward to allow easier application of a sling, positioning of pillows or for dressing.


BedString has 8 handles, alternating right and left side of the string to enable the user to find a good grip.

*Etac BedString with 8 handles on alternating sides of the string.*



## Intended Use

- Support sitting up in bed
- Transfer to bed
- Transfer out of bed

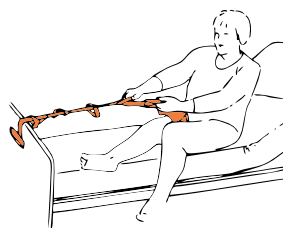

Description	Width x Length		Art.no
	Cm	Inch	
BedString with handles	7,5 x 170	3 x 66.9	IM429

### Specifications

Polyester



### Washing Instructions

Do not use fabric softener or bleach.





# Etac HandyGrip

## Adjustable grip to aid turning in bed

Etac HandyGrip provides help with independent turning in bed. It can also be used when self transferring from wheelchair to bed. Soft, compact and easy to hide away. The adjustable buckle allows the lenght and position of HandyGrip to be varied, providing the user with a good grip at the required position.

*Soft, compact and easy to hide away.*






### Intended Use






- Support independent turning in bed
- Support independent transfer to bed

Description	Width x Length		Art.no
	Cm	Inch	
HandyGrip	5 x 160	2 x 63	IM461

**Specifications**  
Polyester  
Belt Buckle: Acetal and nylon



**Washing Instructions**  
Do not use fabric softener or bleach





# Transfer – Sitting

Tools for seated transfers between bed, wheelchair, chair, toilet, etc.

# Etac Turner

## Patient turner with standing support

Etac Turner is a patient turner with handles that facilitates transfer between seating positions. The caregiver can maintain an ergonomic working position whilst the user is activated and moves to a standing position.

Turner is compact and easy to position when required. The low footplate, only 13 mm high, makes it easy for the user to place their feet on it. The non-slip surface on the footplate offers additional security. The only adjustment necessary is the height of the leg support and the handle. When ready, the transfer can be performed safely whilst the caregiver and user can maintain eye contact and communication.

*Etac Turner is a practical tool which facilitates transfer between seating positions.*



*Turner is available in 2 models, Oval handle for horizontal grip or U-handle for vertical grip.*

## Intended Use

- Sit to stand transfer platform
- Transfer bed to wheelchair / shower chair
- Transfer wheelchair to toilet
- Moving back / forward in chair / wheelchair
- Transfer chair to wheelchair / shower chair
- Support mobilization, standing up / sitting down

Description	Diameter		Art.no
	Cm	Inch	
Turner Oval handle	41	16.1	16090104
Turner U-handle	41	16.1	16090105

### Washing Instructions

Can be wiped off with a soft cloth and mild detergent. Do not use hard tools to clean the bottom plate. Do not use solvents. For disinfection, use 70 % alcohol.



### Specifications

Powder coated steel  
Weight: 14,8 kg / 32.6 lbs





# Etac Transfer Boards

Mind the gap...

A transfer board is intended to bridge shorter gaps when performing seated transfers. The board's low friction upper surface allows the user to transfer independently or with assistance. Transfer boards are used when transferring between bed and wheelchair / shower chair, between wheelchair / chair / toilet chair or to and from the car.

All Etac Transfer Boards have a non-slip surface on the underside. Combining a transfer board with a glide cushion provides even lower friction and is recommended when the transfer involves bare skin, for instance to and from a shower chair or toilet.

## Etac 3B-Board

Etac 3B-Board is a stable and simple transfer board with a traditional curved shape. The board can be placed either curve forward or curve backward depending on the transfer situation and surrounding obstacles, eg: the wheel of the wheelchair. 3B-Board's length allows transfer across wider gaps.

*Simple and traditional board that allows transfer across wider gaps.*



## Etac E-Board

Etac E-Board is a thin and flexible transfer board that allows easier application and removal. Available in 2 models and 4 sizes:

E-Board Wings have wings along the sides, which can be bent up or down. This provides more sturdiness and stability, and the upward wing is used to protect against the wheel of the wheelchair. Available in 2 lengths, 60 and 75 cm.

E-Board Compact is smaller and lighter. As it is easily carried, it is suitable for the more active and independent user. Available in 2 lengths, 60 and 45 cm. The 45 cm board is more suitable for active and independent users, while the 60 cm boards suits the most common transfer situations. The 75 cm board is suitable for longer transfer, eg. in and out of a car.



*Etac E-Board with wings.*



*Etac E-Board without wings.*

## Etac Board

Etac Molift Board is a rectangular transfer board with wings, and available in 2 lengths 60 and 75 cm. The shorter alternative, 60 cm, suits the most common seated transfer situations. The longer alternative, 75 cm, is thicker (8mm) thus providing extra stability for longer transfers such as in and out of a car.



*Molift Board 75 cm is ideally suited for transfer in and out of a car.*

## Etac Butterfly

Etac Butterfly is a long, wide and stable transfer board. It has cut outs for positioning against the wheel of the wheelchair and non-slip pads on the underside, which allows for good positioning and help prevent sliding out of place. Etac Butterfly has a larger sitting and sliding area, which is helpful when the transfer needs to be made in short steps.

Etac Butterfly can be used with the special Butterfly GlideWing, which is mounted on the board. The glide wing slides along the board and provides additional slide function during the entire transfer.



Etac Butterfly with glide wing



## Etac Dolphin

Etac Dolphin is a sturdy and stable transfer board uniquely designed for transfer to the toilet. The board has cut outs for positioning against the wheel of the wheelchair and non-slip pads on the underside, which allows good positioning and avoids sliding out of place. The cut out allows the user to remain on the board while on the toilet.




Etac Dolphin uniquely designed for transfer to toilet.



### Intended Use

- Transfer from bed to wheelchair / shower chair
- Transfer in and out of the car
- Transfer between wheelchair, chair, toilet, shower chair, commode

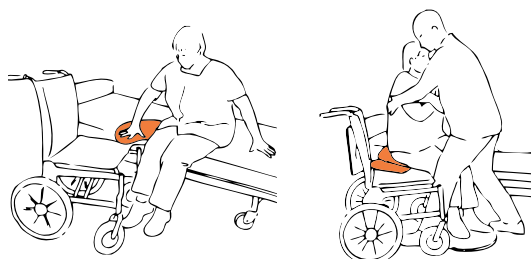

#### Description

	Width x Length x Height		Art.no
	Cm	Inch	
3B-Board, curved transfer board white	23 x 77	9.1 x 30.3	IM403
E-Board Small, transfer board black	20 x 45 x 0,5	7.9 x 17.7 x 0.2	IM410
E-Board Medium, transfer board black	25 x 60 x 0,5	9.8 x 23.6 x 0.2	IM408
E-Board Wide, transfer board wings black	33 x 60 x 0,5	13 x 23 x 0.2	IM404
E-Board Long, transfer board wings black	33 x 75 x 0,6	13 x 29.5 x 0.2	IM405
Etac Board, black	33 x 60 x 0,5	13 x 23 x 0.2	MIM103060
Etac Board XL, black	33 x 75 x 0,8	13 x 29.5 x 0.3	MIM103070
Butterfly, transfer board	32 x 66	12.6 x 26	IM401
Butterfly Wing L	40 x 31	15.7 x 12.2	IM4001
Butterfly Wing XL	52 x 31	20.5 x 12.2	IM4002
Dolphin Toilet Board, white	Lenght: 69	27.2	IM409

#### Specifications

E-board, Etac Board, 3B-board: Polyethylen  
Butterfly, Dolphin: Fiberglass



#### Washing Instructions

Transfer boards can be cleaned with soapy water or disinfectant.



Etac Butterfly Glide Wing:  
Material Outside: Nylon  
Material Inside: Nylon  
Padding: Polyester  
Wipe off with soapy water or disinfectant, washed as described below.  
Do not use fabric softener or bleach.



# Etac PediTurn

## Swivel cushions for the floor

Etac PediTurn is a swivel cushion that facilitates transfer between sitting positions, such as bed to wheelchair. It is used when the user cannot move their feet during the transfer.

PediTurn is available in 2 alternatives:

- Etac PediTurn SOFT: a soft cushion suitable when the user is not wearing shoes. It has a non-slip surface on both sides to avoid sliding out of position.
- Etac PediTurn HARD: a hard plate suitable when the user is wearing shoes.




PediTurn SOFT



PediTurn HARD

## Intended Use

- Support moving feet during seated transfer
- Transfer between bed and wheelchair
- Transfer between wheelchair, chair, shower chair, toilet, commode
- Transfer in and out of a car

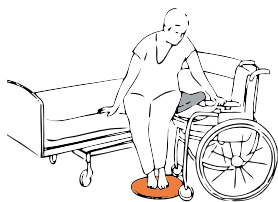

### Description

PediTurn SOFT, swivel cushions for floor, black  
PediTurn HARD, swivel cushions for floor, grey

### Dimensions

Cm	Inch	Art.no
Ø40	Ø15.7	IM99836GT
Ø38	Ø15	IM99700

### Washing Instructions

Can be wiped off with a damp cloth.



### Specifications

PediTurn SOFT:  
Top: Toughtek  
Inside: Nylon  
Underside: Toughtek  
Middle: Polyetylen

PediTurn HARD:  
Material: ABS/TPU  
Inside: HDPE  
Fixing: Brass/steel



# Etac Glide Pads

## Etac Swan

The Etac Swan is an open glide sheet with handles. It is suitable for transfer between bed and wheelchair or in and out of a car and can be used independently or in combination with a transfer board.



*Etac Swan glide pad with handles.*

## Etac HandyTrans


Etac HandyTrans is a short glide pad suitable for transfer between bed and wheelchair or for adjusting position in bed. It can be used independently or in combination with a transfer board. Combining a transfer board and HandyTrans is the optimal solution for a seated transfer in and out of a car.



*Etac HandyTrans gives the caregiver a good grip when transferring on a transfer board.*

### Intended Use

- Support transfer between bed, wheelchair, chair, etc
- Transfer in and out of a car
- Moving back / forward in chair / wheelchair
- Transfer wheelchair to toilet / showerchair

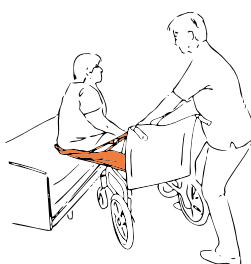

#### Description

Swan, glide pad with handles  
HandyTrans glide pad

#### Width x Length

Cm	Inch
45 x 140	17.7 x 55.1
30 x 40	11.8 x 15.7

#### Art.no

IM2698121
MIM102045

#### Specifications

Material Swan:  
Glide pad: Nylon  
Handles: Polyester  
Padding: Polyester  
Non-slip: Nylon with polyurethan coating

Material HandyTrans:  
Top: Nylon  
Padding: Polyester



#### Washing Instructions Etac Swan

Do not use fabric softener or bleach.



#### Washing Instructions HandyTrans

Autoclave: 130 ° C. Can be cleaned with soapy water or disinfectants. Do not use fabric softener or bleach.





# Transfer – Horizontal

Solutions and tools for horizontal transfers between beds, bed to shower trolley, etc.

# Etac 2Move

## Transfer board for horizontal transfer

Etac 2Move transfer board is used to bridge the gap and even out the height difference between two horizontal positions. It is used for horizontal transfer between beds, bed to shower trolley, bed to x-ray table, stretcher to bed, etc. It is simple to transport and store.

2Move transfer board is available in two lengths; short at 90 or long at 180 cm. The longer model, 180 cm, is available in three variants; narrow foldable, narrow non-foldable and wide foldable. The foldable boards are available in two widths; 50 or 70 cm. The wide foldable board is suitable when the gap between the transfer positions is larger.

2Move is always used in combination with a low friction cover that by sliding around the board minimizes the friction and facilitates the transfer.

Covers are available in four alternatives; low friction nylon, waterproof nylon, disposable or soft padded.




*Etac 2Move transfer board, incl. nylon cover.*



*Disposable cover in roll.*

## Intended Use

- Bridge the gap when transferring between two horizontal positions
- Transfer between beds
- Transfer between bed and shower trolley
- Transfer between bed and stretcher, x-ray table, operating table

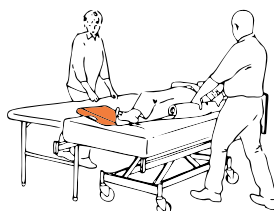

### Description

#### Set

2Move Large Std Set – includes IM8050180 + IM4044  
2Move Small Std Set – includes IM8050090 + IM4045

2Move Transfer Board, Small

2Move Transfer Board, Large foldable

2Move Transfer Board, Large non-foldable

2Move Transfer Board, X-Large foldable

#### Covers

2Move Nylon Cover for IM8050090

2Move Nylon Cover, waterproof, for IM8050090

2Move Disposable Cover for IM8050090, roll with 150 pcs

2Move Nylon Cover for IM8050180 & IM8150180

2Move Nylon Cover, waterproof, for IM8050180 & IM8150180

2Move Disposable Cover for IM8050180 & IM8150180, roll with 75 pcs

2Move Nylon Cover, waterproof for IM8070180

TransferMattress, soft padded cover, for IM8050180 + IM8150180

### Width x Length

Cm      Inch

See below

See below

50 x 90

50 x 180

50 x 180

70 x 180

19.7 x 35.4

19.7 x 70.9

19.7 x 70.9

19.7 x 70.9

19.7 x 35.4

19.7 x 35.4

23.6 x 39.4

19.7 x 70.9

19.7 x 70.9

23.6 x 78.7

27.6 x 70.9

23.6 x 76.8

### Art.no

IM4307

IM4308

IM8050090

IM8050180

IM8150180

IM8070180

IM4043

IM4045

IM4031

IM4042

IM4044

IM4030

IM4046

IM60/195

### Specifications

2Move, transfer board:

Top: Nylon

Inside: Plywood

Handles: Polyester

Cover: Nylon

Disposable cover: Polyetylen



### Washing Instructions

2Move, transfer board: Can be wiped off with disinfectant.

Nylon cover: Wash as instructions. Do not use fabric softener or bleach



# Etac TransferMattress

## Soft padded mattress for horizontal transfer

Etac TransferMattress facilitates horizontal transfer between two lying positions, such as between beds, bed to shower trolley, bed to x-ray table, etc. It can also be used for turning in bed, for instance from “face up” to “face down”. Transfer Mattress is made of a soft padded cotton/polyester material and available in two widths, 60 cm and 70 cm.

Transfer Mattress can be combined with a waterproof nylon cover or disposable cover. Transfer Mattress should not be used if there is a gap between the two transfer surfaces. Etac TransferMattress facilitates horizontal transfer between two lying positions, such as between beds, bed to shower trolley, bed to x-ray table, etc. It can also be used for turning in bed, for instance from “face up” to “face down”. Transfer Mattress is made of a soft padded cotton/polyester material and available in two widths, 60 cm and 70 cm.


Transfer Mattress can be combined with a waterproof nylon cover or disposable cover. Transfer Mattress should not be used if there is a gap between the two transfer surfaces.





The padded mattress provides a soft and comfortable horizontal transfer.

### Intended Use

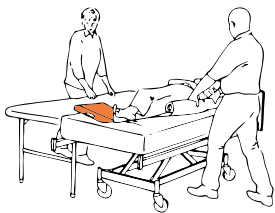- Transfer between beds
- Transfer between bed and shower trolley
- Transfer between bed and stretcher, x-ray table, operating table
- Turning in bed



Description	Width x Length		Art.no	<b>Specifications</b> Outside: 65 % polyester/35 % cotton Padding: Polyester Inside: Nylon  
	Cm	Inch		
<b>Set</b>				
TransferMattress Narrow Set – includes IM60/195 + IM60220N			IM4304	
TransferMattress Wide Set – includes IM70/195 + IM7000N			IM4305	
TransferMattress, narrow	60 x 195	23.6 x 77.6	IM60/195	
TransferMattress, wide	70 x 195	27.6 x 77.6	IM70/195	
<b>Accessories</b>				<b>Washing Instructions</b> Do not use fabric softener or bleach.  
Nylon cover for IM60/195	60 x 220	23.6 x 86.6	IM60220N	
Nylon cover for IM70/195	70 x 200	27.6 x 78.7	IM7000N	
Disposable cover, roll with 75 pcs	60 x 200	23.6 x 78.7	IM4030	









# Positioning – Sitting

Tools for positioning when sitting.

# Etac OneWayGlide

OneWayGlide is a cushion that allows sliding easily in one direction whilst resisting sliding in the other direction. It is suitable for users that slide forward when sitting. OneWayGlide facilitates independent or assisted positioning back in the chair whilst helping to prevent the user from sliding back forward again.

## Etac OneWayGlide

Available in 3 variants; OneWayGlide non-slip, OneWayGlide velour, OneWayGlide pad. OneWayGlide non-slip is a tubular cushion with a non-slip outer material that grips to any surface. It can be combined with a cotton cover when necessary.

OneWayGlide velour is a tubular cushion with velour material. More comfortable to sit on, but needs to be used on a textile surface. OneWayGlide pad is a one layer pad with velour top suitable for use directly on a textile surface.



OneWayGlide velour tubular cushion

OneWayGlide non-slip tubular cushion

OneWayGlide pad velour

## Etac OneWayGlide Long

Etac OneWayGlide Long is a long and open cushion with handles and velour top surface. The length allows for re-positioning to be performed several times in one application. The handles provide a good grip, allowing the caregiver to assist. Available in two variants; with or without tie strings. The tie strings are used to tie the cushion to the wheelchair frame when necessary.



Etac OneWayGlide Long with handles and tie strings.

### Intended Use

- Positioning back in chair / wheelchair / day chair



Description	Width x Length		Art.no	Specifications
	Cm	Inch		
OneWayGlide, tube non-slip	37 x 43	14.6 x 16.9	IM39/8	Padding: Polyester Inside: Nylon OneWay material: Polyester Top: Antislip Velour model: Top: Polyester, Nylon   <b>Washing Instructions</b> Do not use fabric softener or bleach.  
OneWayGlide, tube non-slip	45 x 45	17.7 x 17.7	IM38/8	
OneWayGlide, tube non-slip	80 x 60	31.5 x 23.6	IM37/8	
OneWayGlide, tube velour	37 x 43	14.6 x 16.9	IM45/8	
OneWayGlide, tube, velour	40 x 50	15.7 x 19.7	IM50/8	
OneWayGlide, pad, velour	43 x 37	16.9 x 14.6	IM44/8	
OneWayGlide Long, velour, tie strings	40 x 130	15.7 x 51.2	IM33/8	
OneWayGlide Long	40 x 130	15.7 x 51.2	IM34/8	
Cover for IM39/8	39 x 45	15.4 x 17.7	IM391	
Cover for IM38/8	47 x 47	18.5 x 18.5	IM381	

# Etac Swivel Cushions

## Etac AutoTurn

The Etac AutoTurn facilitates getting in and out of a car. The low friction allows the user to slide and turn into the desired position in the car seat. This minimizes strain and twisting of feet, knees, hip and back. AutoTurn is fixed around the car seat with a belt and buckle. Handles at the sides can be used to assist the user into place, and small straps at the side can be attached around the seat belt, firstly as a location aid and then secondly as additional security.



Etac AutoTurn

## Etac EasyTurn

Etac EasyTurn is a soft swivel cushion that makes it easier for the user to get in and out of a car or bed.

EasyTurn is easy to apply and use. It can for instance be used in workplaces where the personnel often get in and out of the car.



Etac EasyTurn

## Etac HandySwing Seat

Etac HandySwing Seat is a soft swivel cushion specifically suitable to support turning in and out of bed or chair. HandySwing Seat is softer and foldable which makes it easier to place under the user.



Etac HandySwing Seat

### Intended Use

- In and out of a car
- In and out of bed



#### Description

AutoTurn, swivel cushions for car
EasyTurn, swivel cushions
EasyTurn, swivel cushions
EasyTurn, swivel cushions
HandySwing Seat

#### Dimensions

Cm	Inch
Ø45	17.7
Ø40	15.7
Ø45	17.7
Ø50	19.7
Ø40	15.7

#### Art.no

IM99999
IM999405
IM990454
IM999505
MIM106010

#### Specifications

AutoTurn:  
Top: Velour (polyester)  
Inside: Nylon  
Underside: Nylon with PU-coating

EasyTurn:  
Top: Velour (polyester)  
Inside: Nylon  
Backside: Nylon with polyurethancoating  
Middle: Polyetylen

HandySwing Seat:  
Top: Polyester  
Inside: Nylon  
Underside: Nylon with polyurethancoating

#### Washing Instructions


AutoTurn and EasyTurn:  
Can be wiped with a damp cloth



HandySwing Seat:  
Autoclave: 130 °C.  
Can be wiped off with soapy water or disinfection





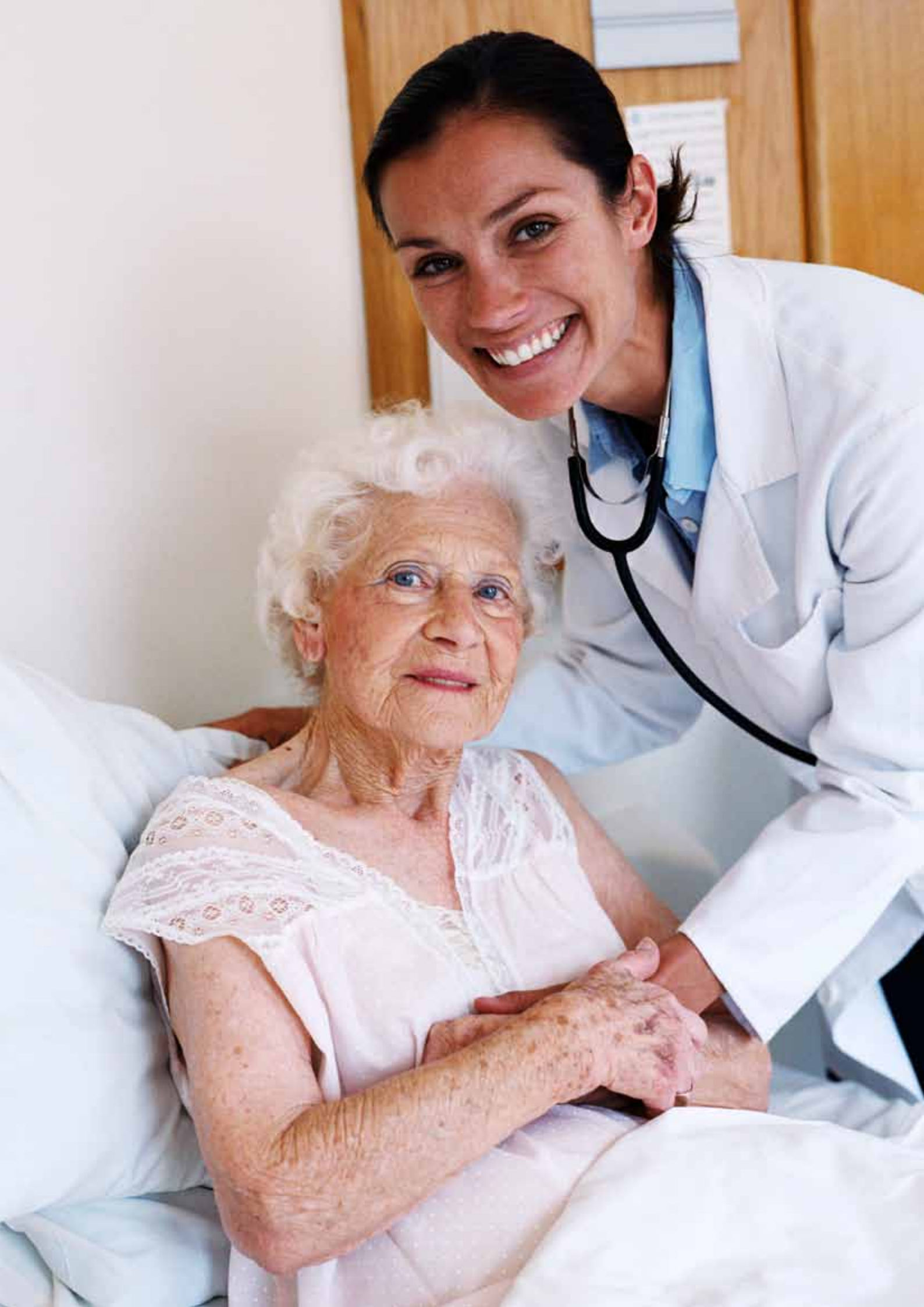



# Positioning – Horizontal

Systems and tools for horizontal positioning, turning, pressure care, moving higher up in bed.



# Etac MultiGlide Glove

## Green fingers

Etac MultiGlide Glove can be used in many different manual handling situations. Using the glove facilitates placing the hands between the user and mattress or cushion. Small re-positioning adjustments, movements and smoothing out the bed sheets are common activities for which the gloves are very useful, and will provide an improved solution for the user and the caregiver. The gloves are also useful for instance when checking the pressure of an air cushion on a wheelchair, whilst the user is still sitting on it.

MultiGlide Gloves are available in two variants: Simple MultiGlide Glove in waterproof nylon and padded MultiGlide Glove.




*Simple MultiGlide Glove in waterproof nylon.*

*Padded MultiGlide Glove*

## Intended Use

- Positioning
- Small re-positioning adjustments
- Hand check of anti-decubitus cushions and mattresses

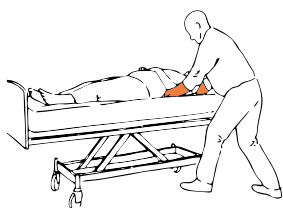

### Description

MultiGlide Glove (1 pair)
MultiGlide Glove Quilted (1 pcs)

### Width x Length

Cm	Inch
20 x 50	7.9 x 19.7
15 x 35	5.9 x 13.8

### Art.no

IM35
IM8000701

### Specifications

MultiGlide Glove, quilted:  
Inside: Nylon  
Outside: Nylon  
Padding: Polyester



MultiGlide Glove, nylon:  
Nylon with polyurethan coating.



### Washing Instructions

Do not use fabric softener or bleach.



# Etac MultiGlide

## Nylon slide sheet with high-low friction

Etac MultiGlide is a thin tubular slide sheet with ultra low friction on the inside and high friction on the outside. The outside surface also provides a waterproof barrier. MultiGlide is suitable when there is a need to minimize friction under the pressure points, for instance when transferring and turning in bed, between bed and wheelchair, on the operating table or x-ray table.

MultiGlide is to be applied when necessary and removed after use. MultiGlide is available in several sizes. Each MultiGlide has a loop strap for easy storage and colour codes for differentiation. MultiGlide should always be removed after use.

Outside: high friction



Inside: ultra low friction



*MultiGlide with high friction outside ensures that it remains in place under the user during transfer and turning.*

## Intended Use

- Minimize friction under the pressure
- Turning
- Positioning
- Higher up in bed
- Bed to bed transfer

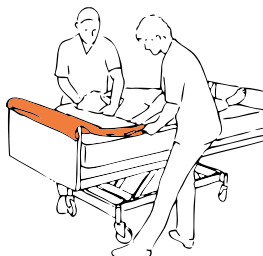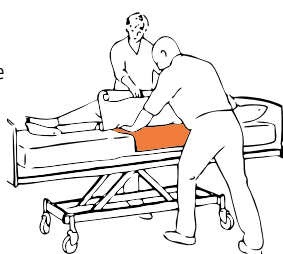

### Description

MultiGlide, waterproof, tubular, blue strap  
 MultiGlide, waterproof, tubular, yellow strap  
 MultiGlide, waterproof, tubular, brown strap  
 MultiGlide, waterproof, tubular, red strap  
 MultiGlide, waterproof, tubular, lightbrown strap  
 MultiGlide, waterproof, tubular, green strap  
 MultiGlide, waterproof, tubular, white strap  
 MultiGlide, waterproof, tubular, green strap  
 MultiGlide, waterproof, tubular, grey strap  
 MultiGlide, waterproof, tubular  
 MultiGlide, waterproof, tubular, pink strap  
 MultiGlide, waterproof, tubular, lightblue strap  
 MultiGlide, waterproof, tubular, purple strap  
 MultiGlide, waterproof, open

### Width x Length

Cm

Inch

40 x 40	15.7 x 15.7	IM40
60 x 60	23.6 x 23.6	IM66
72 x 70	28.3 x 27.6	IM77
72 x 100	27.6 x 39.4	IM80
70 x 200	27.6 x 78.7	IM82
72 x 130	27.6 x 51.2	IM83
100 x 70	39.4 x 27.6	IM100
100 x 135	39.4 x 53.1	IM103/1
108 x 80	42.5 x 31.5	IM108
120 x 70	47.2 x 27.6	IM120
130 x 80	51.2 x 31.5	IM130
160 x 80	63 x 31.5	IM168
200 x 70	78.7 x 27.6	IM200
100 x 140	39.4 x 55.1	IM100/140

### Art.no

### Specifications

Nylon with polyurethan coating



### Washing Instructions

Do not use fabric softener or bleach.  
 Can be wiped with rubbing alcohol.



# Etac MultiGlide SG

## Nylon slide sheet with low-low friction

Etac MultiGlide SG is suitable for all types of situations when there is a necessity to minimize friction under pressure points. It is ultra low friction both inside and outside with the double layer minimizing friction to a maximum.

MultiGlide SG is easy to insert and remove, and should always be removed after use. MultiGlide SG tubular is available in several sizes and also in a 10 metre roll that can be cut to any desired length. MultiGlide SG tubular has a black loop strap for easy storage and differentiation. MultiGlide SG should always be removed after use.

Outside: ultra low friction

Inside: ultra low friction


*MultiGlide SG low-low friction is easy to apply and remove.*



*MultiGlide SG roll that can be cut down to any desired length.*

## Intended Use

- Minimize friction under the pressure points
- Turning
- Positioning
- Higher up in bed
- Bed to bed transfer
- Dressing

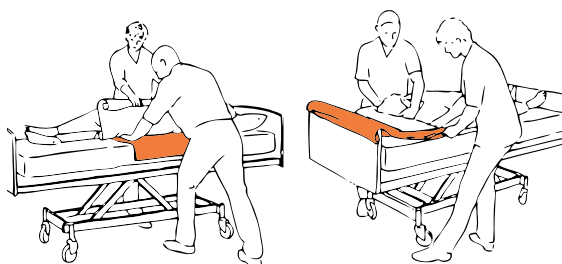

### Description

MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, black strap  
 MultiGlide SG, tubular nylon sheet, open  
 MultiGlide SG, nylon sheet, on roll

### Width x Length

Cm	Inch
40 x 40	15.7 x 15.7
60 x 60	23.6 x 23.6
72 x 70	28.3 x 27.6
72 x 100	28.3 x 39.4
70 x 200	27.6 x 78.7
72 x 130	28.3 x 51.2
100 x 70	39.4 x 27.6
108 x 80	42.5 x 31.5
120 x 90	47.2 x 35.4
130 x 80	51.2 x 31.5
200 x 70	78.7 x 27.6
150 x 70	59.1 x 27.6
150 x 1000	59.1 x 293.7

### Art.no

IM0040  
 IM0066  
 IM0077  
 IM0080  
 IM0082  
 IM0083  
 IM00100  
 IM00108  
 IM00120/1  
 IM00130  
 IM00200  
 IM00150/70  
 IM2245

### Specifications

Nylon



### Washing Instructions

Can be wiped with rubbing alcohol. Do not use fabric softener or bleach.



# Etac MultiGlide with handles

Nylon sheet high-low friction with integrated handles

Etac MutliGlide with handles is a tubular slide sheet with high friction on the outside surface and ultra low friction inside. It is suitable for all types of situations when there is a necessity to minimize friction under pressure points.

MultiGlide with handles is specifically designed to facilitate transfers up and down in bed, x-ray table or MT-scanner. It is easy to insert and remove, and should always be removed after use.



Outside: high friction  
Inside: ultra low friction



Etac MultiGlide with integrated handles

## Intended Use

- Minimize friction under the pressure points
- Higher up in bed
- Turning
- Positioning
- Bed to bed transfer

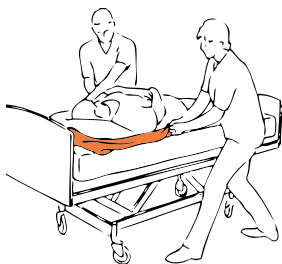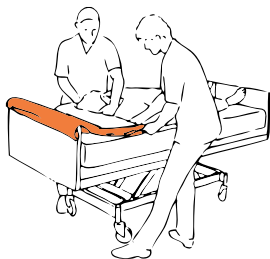

### Description

MultiGlide with handles, waterproof, pink strap  
MultiGlide with handles, waterproof, lightblue strap

### Width x Length

Cm	Inch	Art.no
130 x 80	51.2 x 31.5	IM130DR
160 x 80	63 x 31.5	IM160DR

### Specifications

Nylon with polyurethancoating  
Handles: Polyester



### Washing Instructions

Do not use fabric softener or bleach.



# Etac GlideCushion

## Padded glide cushions

Etac GlideCushions are used for turning in bed, moving up or down a bed and transfer from bed to wheelchair. It can be used to support independent movement or to aid assisted movement. It is available in two variants; GlideCushion nylon top or GlideCushion cotton/polyester top, which can be combined with a nylon or disposable cover.



*Etac GlideCushion with cotton/polyester top.*

# Etac TurningMattress

## Padded turning mattress


Etac TurningMattress is suitable for turning in bed. It is placed in bed and the user remains on top of it. The ultra low friction facilitates a user with reduced mobility to the point that they may be able to independently turn in bed. If necessary, one or two caregivers may assist the turning.

*Etac TurningMattress with cotton/polyester top.*



## Intended Use

- Minimize friction under pressure points
- Move higher up in bed
- Turn in bed, either independently or assisted

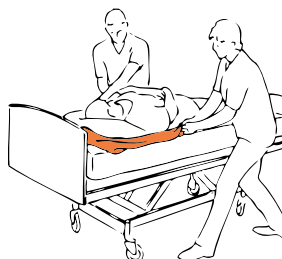

### Description

**Width x Length**  
Cm      Inch

**Art.no**

#### Set

GlideCushion X-Small Set - includes IM60/50 + IM6000N      IM4303

GlideCushion, nylon	60 x 25	23.6 x 9.8	IM60/25N
GlideCushion, polyester/cotton	60 x 50	23.6 x 19.7	IM60/50
GlideCushion, polyester/cotton	60 x 90	23.6 x 35.4	IM60/90
GlideCushion, polyester/cotton	69 x 70	27.2 x 27.6	IM69/70
GlideCushion, polyester/cotton	90 x 90	25.4 x 35.4	IM90/90
TurningMattress polyester/cotton	92 x 195	36.2 x 76.8	IM92/195

#### Accessories

Nylon cover, waterproof for IM60/50	60 x 60	23.6 x 23.6	IM6000N
Disposable cover, roll with 150 pcs.	60 x 100	23.6 x 39.4	IM4031

### Specifications GlideCushion

Outside: 65 % polyester/35 % cotton  
Inside: Nylon  
Padding: Polyester

Model with Outside in nylon:

Padding: Polyester  
Outside: Nylon



### Washing Instructions

Do not use fabric softener or bleach.





# Etac SatinSheet System

A complete in bed system for turning and positioning

Etac SatinSheet System is developed to cover many different levels of need. The system can be used for everyone from a user with slightly reduced/impaired mobility that may be able to turn independently in bed, to a user with high need of caregiver assistance. When the need is increased, the base satin sheet can be combined with alternative draw sheets, whether that be a 2-direction or a 4-direction solution.

## SatinSheet 2Direction

SatinSheet 2D has a low-friction satin area in the centre of the sheet that allows movement from side-to-side thus facilitating turning in bed. The edges of the sheet are of cotton/polyester and will brake movement, allowing the user to sit on the edge of the bed without slipping.

SatinSheet 2D base sheet is available in 3 lengths; Mini provides low friction under the hip section, Midi covers also the shoulder section and Maxi covers the entire length of the bed. On Midi and Maxi lengths there is a choice of 60 or 70 cm satin width.

SatinSheet 2D Maxi is available in four alternatives;

- Maxi Flat sheet (240x200 or 200x200) is the standard and is tucked in as a normal bed sheet.
- Maxi Corner sheet (90x200 cm) with elastic straps at the corners for fixing to the mattress
- Maxi Fit sheet (90x200 cm) a fitted sheet with locking strings and the lower satin area covers the entire width of the mattress to help with sliding the feet in and out of bed.
- Maxi Double (280x200) a flat sheet for double bed with satin on one side.

*SatinSheet2D available in several versions to suit different needs.*




*SatinSheet 2D Maxi Corner*



*SatinSheet 2D Maxi Fit*

Placing of SatinSheet – 3 different lengths

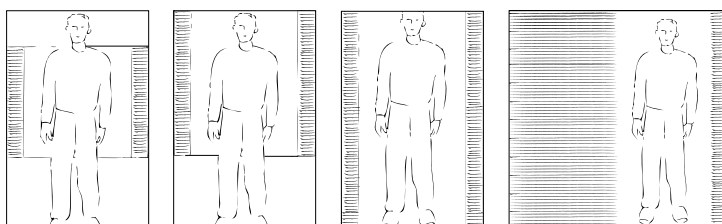

Mini

Midi

Maxi

Double

**DrawSheet – combination for assisted turning**

Combining a SatinSheet 2D base sheet with a DrawSheet, allows the caregiver to assist a user with reduced mobility to turn in bed.

**SatinSheet 4Direction – combination for complete assisted movement**

SatinSheet 4D is a draw sheet with satin on the underside and specifically adapted to be used in combination with a SatinSheet 2D underneath. The combination of a SatinSheet 2D base sheet with a SatinSheet 4D creates a system that allows movement in all directions. This solution is suitable for a user with limited mobility.

**SatinSheet Pillow**

The system includes a pillow case with satin material underside, which can be used for support when moving up or down in bed.

**In2Sheet 4Direction**

The system includes incontinence pads that have satin material on the underside. When used in combination with a SatinSheet 2D it allows for movement in all directions. It has side wings that can be tucked in to lock the sheet in position as well as providing potential grip for the caregiver.



*SatinSheet 2D Maxi corner together with SatinSheet4D.*

**Intended Use**

- In bed system, whether independent or assisted user
- Turning in bed
- Positioning in bed
- Moving higher up in bed

**Description****Width x Length**

Cm Inch

**Art.no****Specifications****Set**

SatinSheet Midi 60 Draw Set – includes IM4113S + IM4118S  
 SatinSheet Mini Draw Set – includes IM4108S + IM4116S  
 SatinSheet Maxi Draw Set – includes IM4114S + IM4118S  
 SatinSheet Maxi FIT 4-D Set – includes IM4112S + IM4115S  
 SatinSheet Midi 4-D Set – includes IM4113S + IM4115S  
 SatinSheet Midi 70 Draw – includes IM4119S+IM4118S

See below  
 See below  
 See below  
 See below  
 See below  
 See below

IM4200S  
 IM4201S  
 IM4202S  
 IM4203S  
 IM4204S  
 IM4205S

Fabric: 60 % cotton/40 % polyester.  
 Satin: Polyester

Specification In2Sheet4Direction: See page 34

**SatinSheet 2Direction – Base Sheet**

SatinSheet Mini 2D (sliding surface B70 x 105 cm)  
 SatinSheet Midi 2D (sliding surface B60 x 140 cm)  
 SatinSheet Midi 2D (sliding surface B70 x 140 cm)  
 SatinSheet Maxi 2D (sliding surface B70 x 200 cm)  
 SatinSheet Maxi 2D Corner (sliding surface B65 x 200 cm)  
 SatinSheet Maxi 2D Fit (sliding surface B70/90 x 200 cm)  
 SatinSheet Double 2D (sliding surface 60x200 cm)

Cm	Inch
195 x 105	76.8 x 41.3
200 x 140	78.7 x 55.1
200 x 140	78.7 x 55.1
200 x 200	78.7 x 78.7
85 x 200	33.5 x 78.7
90 x 200	35.4 x 78.7
280 x 200	110.2 x 78.7

IM4108S  
 IM4119S  
 IM4113S  
 IM4114S  
 IM4107S  
 IM4112S  
 IM4122S

**Washing Instructions**

Ironing improves slide function. Do not use fabric softener or bleach.

**SatinSheet – Draw Sheet**

SatinSheet Draw Mini  
 SatinSheet Draw Midi

Cm	Inch
200 x 100	78.7 x 39.4
200 x 140	78.7 x 55.1

IM4116S  
 IM4118S

**SatinSheet 4Direction – Draw Sheet**

SatinSheet 4D Midi (satin B148 x 140 cm)  
 SatinSheet 4D Maxi (satin B148 x 200 cm)  
 In2Sheet 4Direction (satin B70XL85) with wings

Cm	Inch
200 x 140	78.7 x 55.1
200 x 200	78.7 x 78.7
85 x 90	33.5 x 35.4

IM4115S  
 IM4121S  
 IM4110S

**SatinSheet – Pillow case**

SatinSheet Pillow case

Cm	Inch
60 x 60	23.6 x 23.6

IM4117S

# Etac In2Sheet

## Incontinence pad with satin


Etac In2Sheet is an incontinence pad with low friction underside. It is used directly on top of a cotton bed sheet, with the satin low friction surface down. This reduces friction side-to-side, and is therefore suitable for turning an incontinent user with reduced mobility. In2Sheet has handles along the sides for better grip. In2Sheet is used directly on top of a normal cotton sheet.



*Etac In2Sheet low friction underside facilitates turning in bed .*

### Intended Use

- Incontinent user with reduced mobility
- Turning in bed

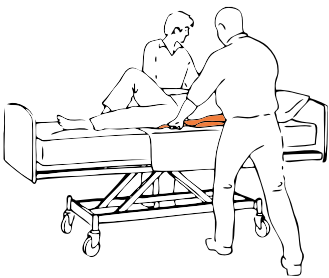

#### Description

In2Sheet incontinence pad with handles (satin B70 x 85)

#### Width x Length

Cm	Inch
90 x 85	35.4 x 33.5

#### Art.no

IM4109

#### Specifications

Top: Polyester  
Absorbent: 65 % polyester/35 % rayon  
Underside: Polyester with PU-coating.  
Satin: Polyester  
Handles: Nylon  
Absorption: 2000ml/m<sup>2</sup>

#### Washing Instructions

Ironing improves slide function.  
Do not use fabric softener or bleach.




# Etac 4WayGlide Mini & Midi

Etac 4WayGlide System consists of two parts; a Nylon Sheet that is fixed to the mattress and Glide Mattress that lies on top.

## Etac 4WayGlide Mini

Etac 4WayGlide Mini is a short (100 cm) in bed system suitable for users with slightly reduced mobility, for instance in the pelvic area. The system facilitates independent or assisted movement. The non-slip stripe along the sides minimizes the risk of the user sliding out of bed.

*Etac 4WayGlide Mini complete system (140 x 100 cm).*

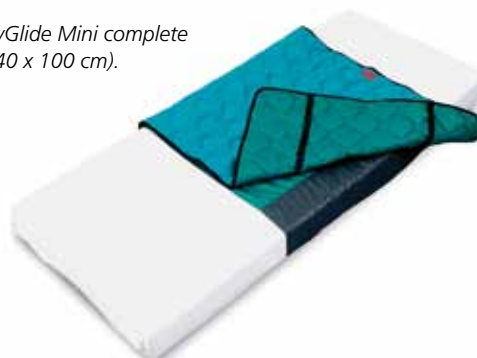

## Etac 4WayGlide Midi


Etac 4WayGlide Midi is a short (150 cm) in bed system suitable for heavy users with independent mobility in the legs. 4WayGlide Midi has a locking system between the nylon sheet and glide mattress and non-slip stripes along the sides.

*Etac 4WayGlide Midi complete system (140 x 150 cm).*



### Intended Use

- In bed system
- Turning, whether independent or assisted
- Positioning
- Moving higher up in bed
- Can be used with a hoist

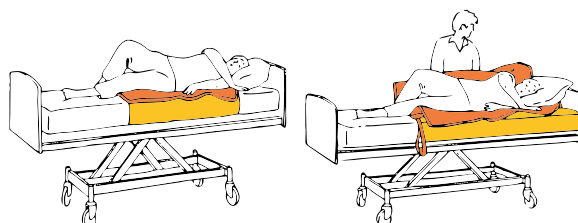

#### Description

##### 4WayGlide Mini

4WayGlide Mini Set – includes IM140/100 + IM140/100NS

4WayGlide Mini Mattress

4WayGlide Mini Nylon sheet with non-slip sides

#### Width x Length

Cm      Inch

140 x 100      55.1 x 39.4

140 x 100      55.1 x 39.4

#### Art. nr

IM4301

IM140/100

IM140/100NS

##### 4WayGlide Midi

4WayGlide Midi Mattress

4WayGlide Midi Nylon sheet with non-slip sides

140 x 150      55.1 x 59.1

140 x 150      55.1 x 59.1

IM140/150

IM140/150NS

#### Accessories

Positioning Wedge with non-slip cover

Positioning Wedge with non-slip cover

#### Width x Height x Length

30 x 20 x 53      11.8 x 7.9 x 20.9

30 x 20 x 80      11.8 x 7.9 x 31.5

IM53NS

IM53/80NS

#### Specifications Mini & Midi

Glide Mattress: Top: Polyester/cotton

Padding: Polyesterfiber

Underside: Nylon

Handles: Polyester



#### Washing Instructions

Do not use fabric softener or bleach.



# Etac 4Way Glide LPL

## Etac 4Way Glide LPL Maxi

Etac 4WayGlide LPL is a complete in bed system for turning and positioning of users with reduced mobility, high weight, pain or pressure issues. 4WayGlide LPL consists of 2 parts; a Glide Mattress and a Nylon Sheet with locking mechanism that is fixed to the mattress. The combination of the underside of the glide mattress and the nylon sheet provides an ultra low friction surface for movement in all directions, which facilitates positioning and turning, whether manual or with support of a hoist.

The locking mechanism on the nylon sheet allows for more control. The non-slip section can be covered or uncovered depending on whether movement or no movement is desired.


The nylon sheet is fixed to the mattress with elastic corner straps, which minimises the risk of “hammocking”. These are available in three different alternatives, see drawings below.

- NylonSheet LPL with NS-lock and NS-sides is the standard and suitable for bed width 80-90 cm
- NylonSheet LPL with NS-lock is also used on bed width 80-90 cm but more suitable when horizontal transfer is necessary
- NylonSheet LPL 3x NS-locks and NS-sides provides more locking possibilities
- NylonSheet LPL Maxi with NS-lock and NS-sides is suitable for wider bed width 100-110 cm




*Etac 4Way Glide*


Three different models of Nylon Sheet.

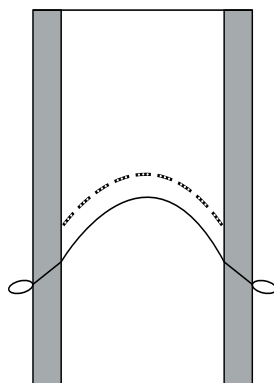

1. Lock with handles  
and non-slip sides

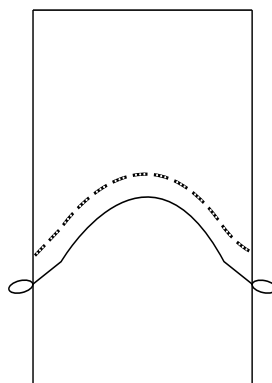

2. Lock with handles

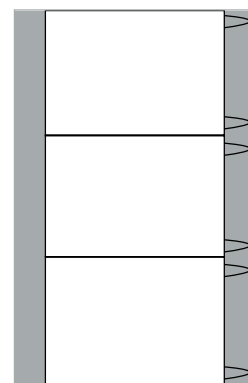

3. 3 x non-slip lock and sides



The Glide Mattres is available in two alternatives for each standard or Maxi system; 4WayGlide Mattress with cotton/polyester top or 4WayGlide Mattress with Polyurethane top. 4WayGlide LPL glide mattress size is 140x200 cm and Maxi size is 180x200 cm, which also has a net reinforcement integrated into the mattress.

- GlideMattress LPL with cotton/polyester top is the standard solution
- GlideMattress LPL with polyurethane top provides a water resistant alternative
- GlideMattress LPL Maxi is a wider alternative for large and heavy patients
- Etac Positioning Wedge can be used to support lying in side position.
- 4WayGlide LPL can be combined with a waterproof (Polyurethane) cover, which further protects the system from fluids (urine, blood, etc.)

*Etac 4WayGlide Mattress with Polyurethane top (140x200 cm or 180x200 cm)*



*Etac 4WayGlide Mattress with cotton/polyester top (140x200cm or 180x200cm)*



## Intended Use

- In bed system, user with reduced mobility
- In bed system, heavy user or user with pain or pressure ulcer
- Turning
- Positioning
- Support horizontal transfer
- Can be used with support of a hoist

### Description

**Width x Length**  
Cm      Inch

**Art.no**

#### Set

4WayGlide LPL Standard Set – includes IM140/200LPL + IM85/200LPLNS

IM4309

4WayGlide LPL 3-Lock Set – includes IM140/200LPL + IM85/2003LPLNS

IM4300

#### 4WayGlide LPL

4WayGlide LPL Mattress, cotton/polyester

140 x 200

55.1 x 78.7

IM140/200LPL

4WayGlide LPL Mattress, polyurethane

140 x 200

55.1 x 78.7

IM140/200LPLG

4WayGlide LPL Nylon Sheet with NS-lock NS-sides

85 x 200

33.5 x 78.7

IM85/200LPLNS

4WayGlide LPL Nylon Sheet with NS-lock

85 x 200

33.5 x 78.7

IM85/200LPL

4WayGlide LPL Nylon Sheet with 3xNS-lock NS-sides

85 x 200

33.5 x 78.7

IM85/2003LPL

#### 4WayGlide LPL Maxi

4WayGlide LPL Maxi Mattress, cotton/polyester

180 x 210

70.9 x 82.7

IM180/200LPL

4WayGlide LPL Maxi Mattress, polyurethane

180 x 200

70.9 x 78.7

IM180/200GLPL

4WayGlide LPL Maxi Nylon Sheet with NS-lock NS-sides

110 x 200

43.3 x 78.7

IM110/200LPLNS

#### Accessories

**Width x Height x Length**

Positioning Wedge with non-slip cover

30 x 20

IM53NS

Positioning Wedge with non-slip cover

30 x 20 x 80

11.8 x 7.9 x 30.5

IM53/80NS

Cover PU/cotton, white

140 x 210

55.1 x 82.7

IM140/200PU

### Specifications LPL and Maxi

Glide Mattress:

Top: Polyester/cotton or polyurethane

Inside: Polyester, net reinforcement - only Maxi

Underside: Nylon (The nylon material is treated for anti-static.)

Handles: Polyester

Padding: Polyester

Nylon Sheet: Nylon

Lock: Nylon with PU-coating



max  
200  
kg

max  
440  
lbs

max  
300  
kg

max  
660  
lbs

LPL

LPL Maxi

### Washing Instructions

Can be wiped with disinfectant (polyurethane top).

Do not use fabric softener or bleach.



# Etac TwinSheet

2-parted glide mattress with integrated incontinence pad

Etac TwinSheet is a 2-part incontinence pad with handles. It is placed on top of an Etac 4WayGlide or similar in bed system. TwinSheet is suitable for turning heavy users with pressure ulcers, users that are difficult to turn and users that are pain sensitive.


TwinSheet can be attached to a hoist for turning of the heavier user, or for maintaining the user on their side during care in bed - hygiene, pressure care, etc.



Etac 4WayGlide with TwinSheet in bed system for care and turning of heavy users with incontinence.

## Intended Use

- In bed incontinence system
- Support the user on their side for hygiene, ulcer care
- Turning
- Positioning
- Can be used with support of a hoist

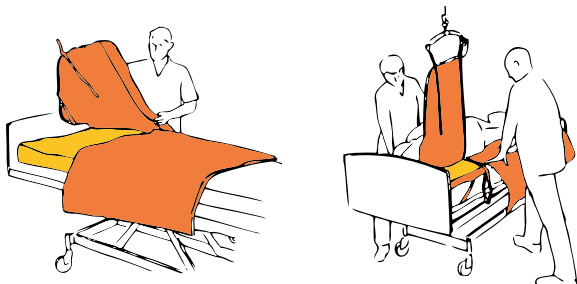

### Description

#### Set

TwinSheet Complete Mattress Set – includes IM150/75 + IM150/85  
TwinSheet Complete 2x Down Set – includes IM150/75 + 2xIM150/85

### Width x Length

Cm	Inch
----	------

### Art.no

TwinSheet Top with absorbing pad	150 x 85	59.1 x 33.5	IM150/75
TwinSheet Down with absorbing pad	150 x 85	59.1 x 33.5	IM150/85
4WayGlide LPL Nylon Sheet with NS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/200LPLNS
4WayGlide LPL Nylon Sheet with NS-lock	85 x 200	33.5 x 78.7	IM85/200LPL
4WayGlide LPL Nylon Sheet with 3xNS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/2003LPL

### Specifications

Top: Polyester  
Middle: 80% polyester/20% viscose  
Protection barrier: Polyurethan  
Bracket: Velcro  
Backside: Polyester, blue  
Handles: Polyester  
Absorption: 2000ml/m²



### Washing Instructions

Do not use fabric softener or bleach.



# Etac TwinSheet 4Glide

## 2-part glide mattress with integrated incontinence pad

Etac TwinSheet 4Glide is a 2-part glide mattress suitable for turning and positioning of incontinent users with more severely reduced mobility. Suitable also for users with dementia, or for those who feel unsafe, have brain damage, pain, pressure ulcers or are heavy.

Turning can be done either manually or with support of a hoist. The handles/straps on the upper part of the glide mattress can be connected to the hoist's spreader bar and, by raising the hoist, assist in turning the user on their side and eventually help maintain the user in this position. Disconnecting the lower part in this position provides better access for care of pressure ulcers, hygiene, dressing and even to change the bedding. The lower part can be ordered separately for replacement during washing.

A complete TwinSheet 4Glide system consists of two parts:


- TwinSheet 4Glide mattress
- Nylon sheet to be fixed to the bed mattress, see page 36 4WayGlide LPL System for the different models available.



Complete 2-part TwinSheet 4Glide system including nylon sheet

## Intended Use

- In bed system with integrated incontinence pad
- Turning
- Support remaining in the side position
- Personal hygiene, ulcer care and dressing
- Positioning
- Moving higher up in bed
- Horizontal transfer
- Can be used in combination with a hoist

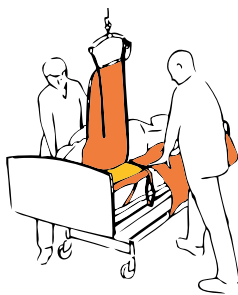

### Description

TwinSheet 4Glide Top part	150 x 75	59.1 x 29.5	IM150/75N
TwinSheet 4Glide Down part	150 x 140	59.1 x 55.1	IM150/140N
4WayGlide LPL Nylon Sheet with NS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/200LPLNS
4WayGlide LPL Nylon Sheet with NS-lock	85 x 200	33.5 x 78.7	IM85/200LPL
4WayGlide LPL Nylon Sheet with 3xNS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/2003LPL

### Width x Length

Cm	Inch
150 x 75	59.1 x 29.5
150 x 140	59.1 x 55.1
85 x 200	33.5 x 78.7
85 x 200	33.5 x 78.7
85 x 200	33.5 x 78.7

### Art.no

IM150/75N
IM150/140N
IM85/200LPLNS
IM85/200LPL
IM85/2003LPL

### Specifications

Glide mattress:  
 Top in microfiber: Polyester  
 Middle: 80% polyester/20% viscose  
 Cover: Polyurethan  
 Backside: Nylon  
 Handles: Polyester  
 Absorption: 2000ml/m<sup>2</sup>

### Accessories

Positioning Wedge non-slip cover
Positioning Wedge non-slip cover

### Width x Height x Length

30 x 20 x 53	11.8 x 7.9 x 20.9	IM53NS
30 x 20 x 80	11.8 x 7.9 x 31.5	IM53/80NS

### Washing Instructions

Do not use fabric softener or bleach.



Nylon Sheet: Nylon  
 Locking system: Polyurethan-covered nylon

# Etac AirGlide LPL

## Positioning system with integrated pressure relieving mattress

Etac AirGlide is a pressure relieving air mattress integrated with a complete in bed positioning system for all positioning and turning. AirGlide is suitable for users that are at risk of or already have pressure ulcers up to stage 2.

AirGlide LPL is a unique system that combines the benefits of an in bed positioning system (Etac 4WayGlide LPL) with a pressure relieving air mattress (Repose). Repose air mattress is made of a thermoplastic membrane that allows optimal pressure relief, which in turn minimizes the risk of pressure ulcers. The surface is soft, skin-friendly and allows the skin to breathe. It is easy to clean/disinfect.


AirGlide LPL facilitates positioning in all directions and it is easy to put in place and use.





AirGlide LPL, unique pressure relieving and integrated positioning system.

### Intended Use

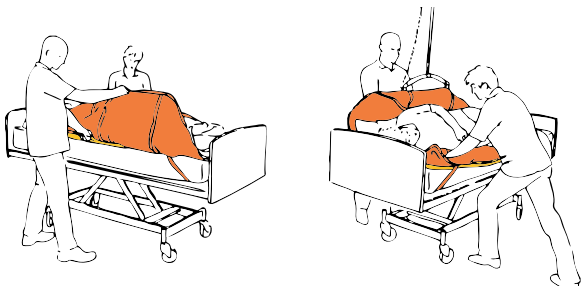- In bed positioning system with integrated pressure relieving mattress
- Users at risk of or with pressure ulcer
- Turning
- Positioning
- Moving higher up in bed
- Horizontal transfer
- Can be used in combination with a hoist



Description	Dimensions		Art.no	
	Cm	Inch		
<b>Set</b> AirGlide Complete Set – includes IM120017, IM85/200LPLNS, IM78000LPL, IM78200, 2x IM78300			IM4311	<b>Washing Instructions</b>  Do not use fabric softener or bleach Cotton Sheet: Se symbols. Repose air mattress: soapy water / detergent diluted in water according to hospital routines for mattress cleaning.  The material can be cleaned with: Hycolin 1,5%, Clearcoc 1:10, Milton 1:10, Hypondlorite 1:100, Sterilot 2%, Chlorhexidin 1:200 (i 70% Isopropyl).  
AirGlide Repose Air Mattress with pump	B140 x 190	55.1 x 74.8	IM78200	
AirGlide LPL Nylon Sledge	B140 x 190	55.1 x 74.8	IM78000LPL	
AirGlide Cotton Cover (2 pcs)	B140 x 190	55.1 x 74.8	IM78300	
4WayGlide LPL Nylon Sheet with NS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/200LPLNS	
<b>Accessories</b> Etac Bag, black carry bag			IM120017	
AirGlide Repair kit			IM78250	
4WayGlide LPL Nylon Sheet with NS-lock	85 x 200	33.5 x 78.7	IM85/200LPL	
4WayGlide LPL Nylon Sheet with 3xNS-lock NS-sides	85 x 200	33.5 x 78.7	IM85/2003LPL	
<b>Specifications</b> Sheet: Nylon with non-slip/PU-coated nylon, AirGlide air mattress: Termoplastic polyurethan, Cover: Cotton				





# Etac OneManSling

The single carer’s companion

Etac OneManSling makes it possible for one caregiver to move a user higher up in bed. Fasten the OneManSling using the loop at one side of the head end of the bed and by using the leverage created by the sling and the sliding cushion, using minimal effort, move the user higher up in bed. To minimise friction, place a slide sheet under the user’s pressure points, primarily shoulders and hip section.


The construction of OneManSling, with a strap and a slide cushion, together with a MultiGlide slide sheet, gives very good leverage for an effortless movement.



*Etac OneManSling makes it possible for one caregiver to move a user higher up in bed.*

## Intended Use

- Assisted move higher up in bed
- Assisted move legs in and out of bed

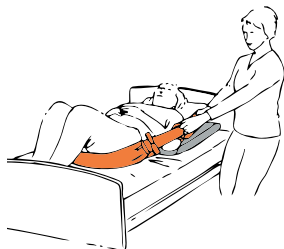

Description	Width x Length		Art.no
	Cm	Inch	
OneManSling, 6 handles	7,5 x 300	3 x 118.1	IM460PU

**Specifications**  
Strap: Polyester  
Glide cushion inside: Nylon  
Padding: Polyester  
Outside: Polyurethan

CE

max

150

kg

max

330

lbs

**Washing Instructions**  
Do not use fabric softener or bleach.

# Etac Positioning Wedge

Support and positioning cushions with non-slip material

Etac Positioning Wedge supports the user on their side after turning, when resting or when performing hygiene or wound care, thus replacing having a caregiver in a static holding position.


Positioning Wedges can be combined with Etac 4WayGlide System, SatinSheet System, Multi-Glide or other slide sheets. The cover is a non-slip Polyurethane material that helps to keep the Wedge in place independent of surface material.



*The Etac Positioning Wedges have unequally angled sides, turn the wedge around and use the angle that fits best for the situation.*

## Intended Use

- Support static positioning
- Resting
- Care and personal hygiene
- Can be used alone or in combination with other positioning tools

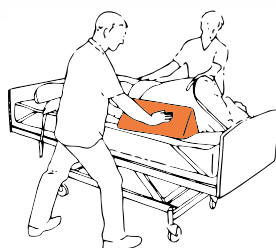

### Description

Positioning Wedge Small  
Positioning Wedge Large  
Cover for Positioning Wedge Small  
Cover for Positioning Wedge Large

### Width x Height x Length

Cm	Inch
30 x 20 x 53	11.8 x 7.8 x 20.9
30 x 20 x 80	11.8 x 7.8 x 31.5
30 x 20 x 53	11.8 x 7.8 x 20.9
30 x 20 x 80	11.8 x 7.8 x 31.5

### Art.no

IM53NS  
IM53/80NS  
IM5300NS  
IM5300/80NS

### Specifications

Soft foam  
Cover: Nylon with PU-coating, zipper

### Washing Instructions

Can be wiped with disinfectant  
Do not use fabric softener or bleach.







# Special Solutions

Solutions and tools for specific situations and needs, hospital environment and tight spaces.



# Etac SlingOn

Multi-tool for application of lifting sling and positioning

Etac SlingOn is primarily designed for easy insertion of a hoist sling in bed without the need for rolling or moving the user. SlingOn consists of two separate MultiGlide ultra low friction nylon slide sheets held together by snap buttons in the corners and handles along the sides. This holds the slide sheets well in place and facilitates application. Etac SlingOn is an optimal tool for users that needs frequent hoisting, heavy users, or users that may not be rolled for application of a sling. And since the material is the same as used for the MultiGlide slide sheets, it can also be used for all positioning in bed.

SlingOn is put in place under the user using a simple folding technique, whether for application of sling or for positioning. The hoist sling is then inserted between the two layers of MultiGlide. The ultra low friction facilitates sliding the sling in place underneath the user, and adjusted if necessary. When the lifting sling is in place, remove the top part of SlingOn.



Note: The user is always lifted in the sling, never in SlingOn! Always remove the top part of SlingOn before hoisting!



*Etac SlingOn is the optimal solution for insertion of a hoist sling without rolling.*

## Intended Use

- Application of a hoist sling without rolling
- Minimise friction under pressure points
- Turning
- Positioning
- Move higher up in bed

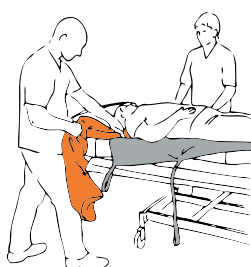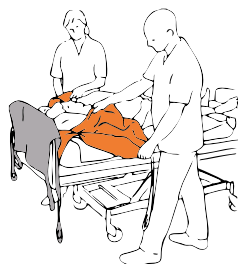

### Description

SlingOn Large  
SlingOn Medium  
SlingOn Small

### Width x Length

Cm	Inch
120 x 150	47.2 x 59.1
80 x 100	31.5 x 39.4
80 x 170	31.5 x 66.9

### Art.no

IM437  
IM438  
IM437/S

### Specifications

Nylon  
Strap: Polyester  
Push buttons

### Washing Instructions

Can be wiped with disinfectant.  
Do not use fabric softener or bleach.



# Etac WiseMover

## In bed positioning for hospital environment


Etac WiseMover is a specially developed in bed solution for bedridden patients that require frequent positioning and transfer, for instance patients in an ICU environment. WiseMover stays fixed to the bed; the slide sheets are folded away in the case at the head end and are folded in and out under the patient when required. When rolled out under the patient it can be used for turning, positioning and horizontal transfer. It also facilitates inserting an x-ray cassette under the patient when the patient needs to stay in bed.



*Etac WiseMover can be hidden inside the case at the head end of the bed and be folded in and out under the patient when required.*

## Intended Use

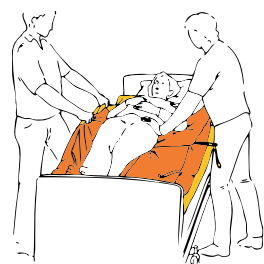

- In bed positioning solution for hospital environment
- Minimise friction under the pressure points
- Turning
- Positioning
- Horizontal transfer
- Inserting x-ray cassette
- Inserting lifting sling



Description	Width x Length		Art.no
	Cm	Inch	
WiseMover transfer system	85 x 200	33.5 x 78.7	IM441







**Specifications**

Surface: Nylon  
Strap: polyester  
Cover: nylon with polyurethan surface

**Washing Instructions**

Do not use fabric softener or bleach.

# Etac TransSeat

Comfortable seat for lifting and transfer

Etac TransSeat is a specially designed seat for lifting and transfer of users where a mechanical hoist cannot be used, for instance while travelling or transferring in and out of an airplane or bus. Several handles along the side, front and back allows for an ergonomic grip.

The user can remain in the TransSeat during the entire trip.



*The user can remain seated in TransSeat during the entire trip and when necessary it can be used to assist transfer at the destination.*

## Intended Use

- Seated transfer when a hoist is unavailable or impractical
- Transfer chair to chair
- Transfer bed to wheelchair
- Transfer in and out of car, airplane, boat, bus, cinema auditorium

### Description

TranSeat JR  
TranSeat SR

### Dimensions

Cm	Inch
B36 x D37 x H40	14.2 x 14.6 x 15.7
B43 x D44 x H40	16.9 x 17.3 x 15.7

### Art.no

MIM002344  
MIM003344

### Specifications

Padding: Polyetylen  
Binding: Polypropylen



### Washing Instructions

Autoclave: 130 °C. Can be wiped off with soapy water or disinfection. Do not use fabric softener or bleach.



# Etac MiniBoard

## Soft transfer stretcher

Etac MiniBoard is suitable for transfer between bed and operating table, turning and positioning in bed and when lifting from the floor. The board in combination with the low friction material facilitates inserting it under the patient.


The 2 variants with head support allow safe and comfortable support for the head during transfer.

*The low friction material facilitates placing Etac MiniBoard under the patient.*



### Intended Use

- Horizontal transfer
- Lift from the floor

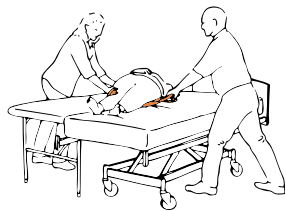

Description	Width x Length		Art.no
	Cm	Inch	
MiniBoard, 4 handles, rectangular	44 X 100	17.3 x 39.4	IM8010044
MiniBoard, 4 handles, rectangular	54 X 110	21.3 x 43.3	IM8011054
MiniBoard, 6 handles, rectangular	54 X 150	21.3 x 59.1	IM8015054
MiniBoard, 7 handles, with head support	54 x 124	21.3 x 48.8	IM8012454
MiniBoard, 11 handles, with head support	54 x 200	21.3 x 78.7	IM8020054

### Specifications

Top and Underside: Nylon  
Inside: Polyetylen  
Handles: Polyester  
Zipper



### Washing Instructions

Can be wiped with disinfectant.  
Remember to remove the stuffing and the plastic sheet before washing.







# X-Ray Solutions

## Etac Thorax SlideSecure

### In bed x-ray system

Etac Thorax SlideSecure facilitates inserting an x-ray cassette under a bedridden patient. It is fixed to the head end of the bed so that it is easily available. Thorax SlideSecure has been developed for bedridden patients that need frequent x-ray examinations.



*Thorax SlideSecure*

## Etac Mole

### Unique glide board for x-ray cassette

Etac Mole is a glide board developed to facilitate placing an x-ray cassette under a patient in bed. Mole consists of two separate boards, held together at the front end so that the x-ray cassette can be placed inside the board. The top board is transparent to assist in controlling the position of the x-ray cassette. Slide the board with x-ray cassette under the patient easily until it is in the desired position.

*Mole X-rayglideboard*



### Intended Use

- Facilitate placing x-ray cassette under patient

### Accessory



## Etac ThoraxSlideSecure bag

Thorax SlideSecure glide bag can be combined with Etac Thorax SlideSecure or Etac Mole for even easier application of the x-ray cassette.

## Etac Scanner ArmSupport

Scanner ArmSupport

### Support and positioning cushion

Etac Scanner ArmSupport is a tool for patients who have difficulty in placing and holding their arms in place during x-ray and scanning. Scanner ArmSupport makes it more comfortable for the patient and minimises risk of movement during the x-ray or scanning to give improved picture quality.

### Intended Use

- Place and support the arms during x-ray and scanning



#### Description

	Dimensions		Art.no
	Cm	Inch	
Thorax SlideSecure	B90 x L80	35.4 x 31.5	IM4008SECURE
Thorax SlideSecure Bag, open 56 cm	B56 x L47	22 x 18.5	IM4017
Mole X-ray Board	B40 x L80	15.7 x 31.5	IM407
Scanner ArmSupport	B30 x L25 x H52	11.8 x 9.8 x 20.5	IM500

#### Specifications

Thorax SlideSecure:  
Nylon with polyurethan coating.



Mole:  
Polyethylen  
Bag: Nylon



Scanner ArmSupport:  
Polyurethanfoam and  
polyurethancover



#### Washing Instructions

Etac Thorax SlideSecure:  
Lock the velcro during washing.  
Do not use fabric softener or bleach.



Etac Mole:  
Can be wiped off with soapy  
water or disinfection



Etac Scanner ArmSupport:  
Can be wiped with disinfectant



# Etac Rescue

## Emergency evacuation slide system

Etac Rescue evacuation slide sheet remains in place under the mattress. In case of emergency, it can be used to quickly transfer a bedridden patient including the entire mattress out of the room and down the stairs. The material is waterproof.



Etac Rescue emergency evacuation slide sheet.



### Intended Use

- Emergency evacuations
- Transfer out of bed, and slide out to safety

Description	Width x Length		Art.no
	Cm	Inch	
Rescue Sheet	75 x 200	29.5 x 78.7	MIM108088




**Specifications**

Material: Nylon  
Strap: Polypropylen  
Buckle: Acetal

**Washing Instructions**

Autoclave: 130 °C.  
Can be wiped off with soapy water or disinfection  
Do not use fabric softener or bleach.





# Etac HipTurner

Special tool for turning and static support in bed

Etac HipTurner was especially designed for turning a patient and supporting them in a static position on their side, while also creating space for placing a bed pan under the patient, for personal hygiene, ulcer care or dressing. HipTurner makes it possible for the caregiver to maintain a good working position thus minimizing strain.


Turning with help of Etac HipTurner can be performed manually or in combination with a hoist.



*Etac HipTurner supports turning and static positioning of the patient on one side.*

## Intended Use

- Turning in bed
- Support static positioning on one side
- Facilitate placing bed pan, personal hygiene, ulcer care or dressing
- Can be used in combination with hoist

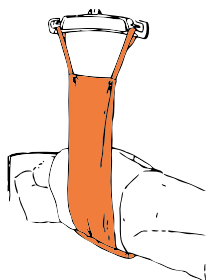


Description	Width x Length x Height		Art.no
	Cm	Inch	
HipTurner (sling B25 x 80 cm)	40 x 20 x 4,5	15.7 x 7.9 x 1.8	IM462

**Specifications**

Nylon  
Strap: Polyester  
Cover: Nylon with PU-coating  
Foam: Polyethylen






max  
200  
kg



max  
440  
lbs

**Washing Instructions**

Do not use fabric softener or bleach. Remove the foam pad during cleaning.



# Etac NonSlip

Non-slip material when increased friction is required

Etac NonSlip Square is a high friction pad that can be used when increased friction is desired. It can for instance be placed under the feet to stay and support the user when moving themselves higher up in bed.

Etac NonSlip Roll is a high friction material on a roll that can be cut to the desired size. Rolls are 150 cm wide and available in two lengths; 5 meters or 30 meters.

*Etac NonSlip Roll*



*Etac NonSlip Square*



## Intended Use

- Increase friction
- Hold feet in place for pushing into position
- Underneath EtacOneWayGlide velour
- Underneath food plate

Description	Width x Length		Art.no
	Cm	Inch	
NonSlip Square	38 x 48	15 x 18.9	IM2248
NonSlip 5 m, grey	150 x 500	59.1 x 196.9	IM2246
NonSlip 30 m, grey	150 x 3000	59.1 x 1181.1	IM2247

### Specifications

Square: Polyethylen  
Non-slip on roll: NonSlip



### Washing Instructions



# Etac CloverFoot

Stool for elevating position of feet

Etac CloverFoot is a stool used to elevate the user’s feet during transfer, for instance when the bed, wheelchair or other seat cannot be adjusted in height. CloverFoot can also be used to elevate and support the user’s feet during toileting and to give a more stable and comfortable position.

Etac CloverFoot has an angled shape front and back so that it can be placed close to any obstacle and to provide the caregiver with more space to maintain a better working position.

*Etac CloverFoot elevating foot stool.*



Intended Use

- Elevate feet for better positioning
- Support underneath feet when the height cannot be adjusted

Description	Width x Height x Lenght		Art.no
	Cm	Inch	
CloverFoot	38 x 9 x 32	15 x 3.5 x 12.6	IM2001

**Specifications**  
Polyethylen

CE

max

150

kg

max

330

lbs

**Washing Instructions**  
CloverFoot can be wiped with disinfectant

# Procedures for delivery and inspection

## First Inspection

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual can be downloaded from [www.etac.com](http://www.etac.com) or contact the local distributor.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

## Periodic Inspection

- Make sure that your Manual Transfer system always is in a perfect condition .
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.
- Lot no/Batch no:

x	x	x	x	x	x	x	x	x	x
Production year		Running number							



### Visual inspection/Check the product

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.



### Mechanical load or stress/Test handles

Pull hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and pull in opposite directions, control the buckle and the stitching.



### Stability test

Try to bend the product and verify that it feels firm, stable and solid.



### Test low/high friction

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the material.

Low friction – slides effortlessly

High friction – no sliding/moving

## Always check

Manual transfer aids with handles, straps and buckles:



Manual transfer aids for sitting and horizontal transfer:



Manual transfer aids with high or low friction:




The product can be scrapped in accordance with national regulations



Read the User Manual

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.

For more information about training and the use of these products, contact the Etac transfer experts – [www.etac.com/exp](http://www.etac.com/exp).

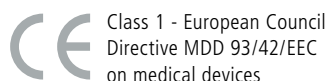

Class 1 - European Council  
Directive MDD 93/42/EEC  
on medical devices











# Explanation Of Symbols





We at Etac continually strive to improve our products. We therefore reserve the right to make changes to the products without prior notice. All measurements shown in sketches or other material are suggestions only, and these products may have faults and deficiencies.

- The products meet the standards of the medical directive for Class 1 products Medical device directive MDD 93/42/EEC
- Warranty: Two years for material and production faults when the product is used correctly.
- Etac's CE certification and Etac's warranty are void when the customer makes any changes, reparations or a combination of these. If in doubt, please contact Etac.
- For more information about Etac's assistive devices, please go to [www.etac.com/exp](http://www.etac.com/exp).
- The product should be disposed of in accordance with national regulations.


## Washing instructions

	Wash/temperature celsius		Ironing, low temperature
	Hand wash		No dry cleaning
	Wiping		Do not bleach
	Tumble drying low temperature		Tumble drying Normal temperature


## Other symbols


	Warning
	Risk of slipping – never leave on the floor
	Risk of gliding in bed
	Batch-nr./Lot-nr.

max 200 kg max 440 lbs Max Weight

 The product should be disposed of in accordance with national regulations.

 Manufacturer

 Class 1 - Medical device  
directive MDD 93/42/EEC

 Read manual

# Product Overview

2Move Transfer board	19
3B-Board	14-15
4Way Glide LPL	36-37
4Way Glide LPL Maxi	36-37
4Way Glide Midi	35
4Way Glide Mini	35
AirGlide LPL	40
AutoTurn	24
BedString	10
Board	14
Butterfly	15
CloverFoot	56
Dolphin toilet board	15
EasyTurn	24
E-Board	14
GlideCushion	31
HandyGrip	11
HandySwing Seat	24
HandyTrans	17
HipTurner	54
In2Sheet	34
MiniBoard	48
MultiGlide SG	29
MultiGlide	28
MultiGlide Glove	27
MultiGlide with handles	30
Non-Sip	55
OneManSling	42
OneWayGlide	23
OneWayGlide long	23
PediTurn	16
Positioning Wedge	43
Rescue Sheet	52
SatinSheet System	32-33
Scanner ArmSupport	51
Sling	8
SlingOn	45
SmartMove	9
SupportBelt	7
Swan	17
Transfer Mattress	20
TransSeat	47
Turner	13
Turning Mattress	31
TwinSheet	38
TwinSheet 4Glide	39
WiseMover	46



"Everyone should be able to live a free and independent life and pursue their dreams regardless of physical circumstances"

Visit us at: [www.etac.com/exp](http://www.etac.com/exp)