

ROMA

MEDICAL



Dalerise

Models 5918 / 5920



Please ensure this manual is read and understood before using the chair

Contents	Page
Introduction	2
Intended Use	2
Technical Specifications	2
Safety Advice	3
Set-Up and Operation	
• Assembly	4
• Power Connection	4
• Motor Connection	5
• Operation	5
Care and Maintenance	5
Warranty	7

Introduction

Congratulations on your purchase of the RMA **Dalerise** rise and recline chair. The chair incorporates a world-patented design to allow the chair to operate fully enclosed for extra safety and efficiency.

Intended Use

The chair is intended to help an adult (maximum weight 175kg) who require the comfort of a fully reclining upholstered chair with the function to aid the user to stand from a seated position.

It is intended to be used in a domestic environment by adults only and is designed for use by one occupant at a time. The chair is designed to conform as much as possible to the natural movements of an average person. Due to the nature of the chair, professional medical advice should be taken if certain movements could aggravate an existing medical condition.

Technical Specifications

Model 5918

Total Height:	142cm
Total Width:	84cm
Total Depth:	76-163cm
Seat Width:	46cm
Seat Depth:	46cm
Seat Height:	51cm
Max. User Weight:	175kg (28st)

Model 5920

Total Height:	142cm
Total Width:	89cm
Total Depth:	76-163cm
Seat Width:	51cm
Seat Depth:	46cm
Seat Height:	51cm
Max. User Weight:	175kg (28st)

Safety Advice



DO:

- Read this manual carefully before assembling and using the chair
- Ensure that the chair is used on a level surface
- Always be aware of persons/objects in your vicinity when operating the chair
- Make sure the backup batteries are replaced if the system is used
- Take care when operating, as powerful motors control the chair
- Unplug mains cables when moving the chair to prevent damage



DO NOT:


- Use by more than one person at any time
- Let children or persons requiring special observation use the chair unattended
- Operate the chair if any cables are damaged
- Use outdoors
- Use in a damp environment such as a shower/bath area
- Sit on the footrest or backrest when in the reclined position
- Sit on the armrests
- Put hands or any objects into any part of the mechanism
- Use the chair if it moves unintentionally. Remove mains power supply and contact your dealer
- Store the chair in a damp environment for any period of time
- Attempt to modify the chair in any way, this will void the warranty

Care should be taken at all times when operating the chair, please follow the advice and instruction set out in this user manual.

Set-up and Operation

Assembly

Your **Dalerise** chair should be delivered in one box, this contains the base frame and the back assembly. Take care when opening the box as sharp objects may damage the upholstery.

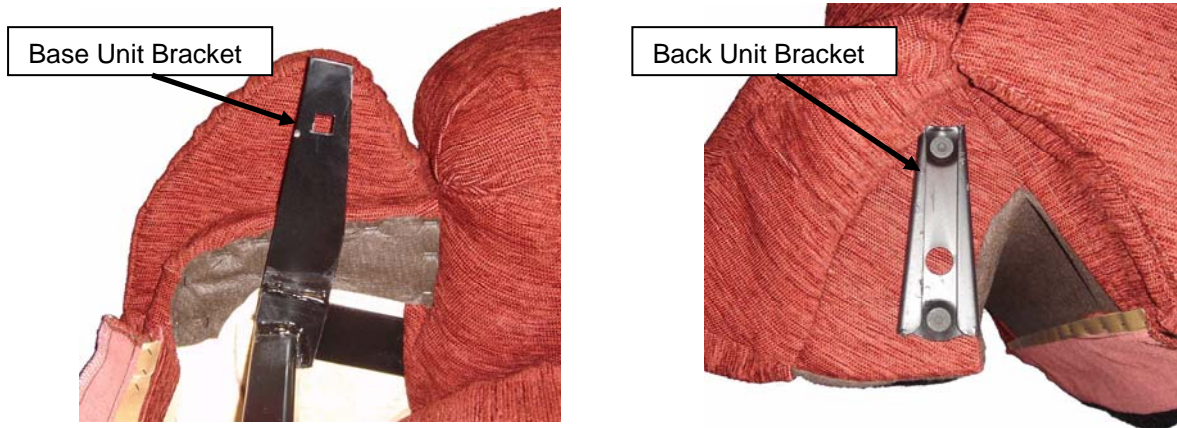


Take the backrest assembly and carefully locate both the base unit brackets inside the back unit brackets. Push the backrest downwards until the base unit brackets are fully located inside the back unit brackets.

Power Connection

Transformer Set-up

Your transformer will need three 9-volt PP3 type batteries (not supplied) for emergency purposes, which will **lower** the chair in the event of power failure. Remove the two screws holding the battery-retaining clip on the underside of the transformer and connect the batteries. Locate the batteries in the slots and refit the battery-retaining clip.

Note: The emergency batteries will only lower the chair once. They must be replaced after being used.

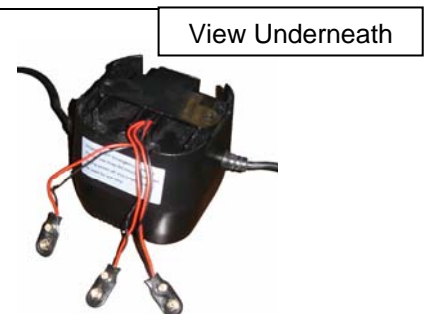

Connecting the Power

At the back of the chair you will see a small lead with a male plug (Y in the picture). Take this lead and connect the female plug (X in picture) from the transformer and push together firmly (a positive 'click' will be felt/heard).

Should the male/female plug not engage properly the chair will not operate due to an insufficient power supply. (If this happens the three batteries under the transformer should be changed immediately, as emergency use will completely drain the power.)

Ensure the back of the chair is positioned at least 40cm away from the wall. Plug the transformer pack into the mains supply and turn on the power. Before sitting in the chair, check that there is sufficient clearance for the backrest and foot supports by fully reclining the chair.



Motor Connections

Each motor has a colour coded connection and is labelled (please see image below).







The handset is connected on the top of the control box as indicated below. Do not force any of the connections. When dis-connecting the plugs, only use the plug body, DO NOT pull on the cable to dis-connect.



Operation

Please see the image opposite which shows the functions of the six buttons on the handset.

Always ensure the chair is in the default 'seat' position before attempting different seating positions.

-  Raise calf support.
-  Lower calf support.
-  Recline backrest.
-  Raise backrest.
-  Seat lift.
-  Seat lower.



Note: The motors on the chair should not be run continuously for more than 2 minutes in every 18. If used continuously, the safety cut out on the transformer will activate. This prevents overheating of the motors. The transformer will automatically reset – this may take up to 30 minutes.

Continued...



Caution! *If the chair is likely to be unattended for some time or if children or persons requiring special supervision have access to the chair, remove the mains plug from the wall and disconnect the transformer. Store the transformer in a safe place.*

Care and Maintenance

Please follow these guidelines to ensure that your **RMA Dalerise** chair will give you reliable service.

The zips fitted to the upholstery are to aid manufacture only. The covers are not designed to be removed for cleaning. Remove small stains with a proprietary stain remover – seek professional advice where required.

Do not attempt to wash the chair as this could cause colours to run and the interior of the chair to become damaged.

You may vacuum the chair using an upholstery attachment.

Take care with rings, buckles, bracelets etc. These can catch on the upholstery and cause damage.

It is not recommended that you attempt to repair any of the rise/recline mechanisms or electronics on the chair. NEVER open any of the motors or the transformer casings. If any cables appear to be damaged replace them immediately.

An authorised RMA dealer using only the approved spare parts must undertake any routine maintenance or repairs.

Replace the emergency batteries every six months or after each time the emergency let down function has been used.

Transportation



Care should be taken when transporting/re-locating your chair.

This should be done by at least two persons due to the weight and dimensions of the chair.

Always lift the chair from the main frame base, NEVER lift the chair by the leg support panel at the front of the chair or backrest as this can damage the chair or cause injury to the persons moving the chair.

Always re-locate the chair on an even surface capable of supporting the chair and an occupant.

WARRANTY TERMS AND CONDITIONS

Standard Terms **(Not transferable)**

1. Metal reclining / inclining mechanism (except as "2" see below) has a limited warranty against defects in material and workmanship for 2 years from date of delivery.
2. Electric motors, electrical components and electrically operated lifting and rolling mechanisms have a limited warranty against defects in materials and workmanship for two (2) years from the date of delivery.
3. All frame parts and springs have a limited warranty against defects in material and workmanship for two (2) years from date of delivery.
4. All other parts (except as "5" & "6" see below) have a limited warranty against defects in materials and workmanship for one (1) year from date of delivery.
5. Padding and foam covered for two (2) years under normal domestic use.

6.Scope of warranty

Warranty coverage applies only to defects in products that are used exclusively for personal, family or household purposes by the original purchaser. It does not cover rental, business, commercial, institutional, or other uses.

If a defect or fault arises, the dealer from where the chair was purchased should be notified immediately.

The manufacturer will not accept liability for damage caused by any misuse, unauthorised alterations or non-observance of the instructions set out in this user manual.

Roma Medical warrants that if a fault occurs within this period due to faulty workmanship or materials that the product will be repaired (or replaced at the Company's discretion) free of charge.

The guarantee excludes parts, which have been subjected to natural wear and tear during this period.

Model No:.....

Serial No:.....

Purchase Date:.....

Dealer's Stamp

Roma Medical Aids Ltd, York Road, Bridgend Industrial Estate,
Bridgend. CF31 3TB
Tel: 01656 674488 Fax: 01656 674499
Email: sales@romamedical.co.uk
www.romamedical.co.uk

RMA reserve the right to change specifications without prior notice