

www.chilterninvadex.co.uk

Showering and Bathing

TransAqua Stainless Steel Shower Chair Range

Mobile showering solutions for the home, nursing home and hospital





Showering and Bathing

TransAqua Stainless Steel Shower Chair Range

Mobile showering solutions for the home, nursing home and hospital

Combining durable stainless steel with modular construction the TransAqua range of shower chairs offer flexibility for both user and carer. A polished stainless steel frame offers resistance to corrosion and ensures a durable, long lasting finish for extended life.



Self Propelled

The TransAqua suits both standard WC's and Clos-o-Mat toilets, reducing storage requirements.

Available in 18", 19" (for Clos-o-Mat only) and 21" model options the TransAqua is designed to suit both user requirements and site conditions. A variety of internal widths can be achieved without the need to select a custom chair.



Attendant Propelled

The new HF welded seat construction is soft, comfortable and completely sealed against water ingress. For storage or transportation the TransAqua can be dismantled without the need to use tools.



HF welded seat

TransAqua Stainless Steel Shower Chair Range

TA18SP - Self Propelled 450mm (18" wide)

TA19SP - Self Propelled 475mm (19" wide) Clos-o-Mat model

TA21SP - Self Propelled 525mm (21" wide)



TA18AP - Attendant Propelled 450mm (18" wide)

TA19AP - Attendant Propelled 475mm (19" wide) Clos-o-Mat model

TA21AP - Attendant Propelled 525mm (21" wide)

Features - All TransAqua models Corrosion resistant stainless steel frame

- HF welded seat
- Swing away and removable locking armrests
- Swing away and removable height adjustable footplates
- Choice of seat styles and sizes
- Removable seat
- Choice of standard accessories to suit user and carer requirements
- Maximum user weight 200kgs (30 stones).
- Clos-o-Mat Compatible models
 (TA19SP / TA19AP)

Self Propelled models

- Anti-tip bars
- Easy to apply brakes
- Choice of 20" (510mm),22" (560mm) and 24" (610mm) wheels
- Rear wheels can be set back for added stability.

Attendant Propelled models

- In-line steering option
- Braked Castor (for added safety when transferring).



Standard Accessories 1. Height adjustable armrests - from 160mm - 275mm height (from the seat) 2. Side pads (1", 2", 3" options to alter internal seating area of chair) 3. Footrests with toe straps 4. Short footrests 5. Footrests with heel 6. 90 degree footrests Footrests with heel/toe straps 8. Waist strap 9. Full support harness 10. Head support 11. Commode pan 12. Cradle 13. Calf strap 14. Foot pad (clip on)

TransAqua Stainless Steel Shower Chair Range Features

New hygienic seat construction

New HF welded seat is completely sealed against water. Seat fabric contains Biocene, an anti-fungal agent that offers increased levels of hygiene with a reduced risk of cross-infection.

Snap on, snap off seat design

Easily removable for cleaning and to help maintain hygiene standards.

Soft padded seat

Moulds to body shape for increased user comfort.



Swing away footrests

Height adjustment

Using leg extension pins the height of the chair can be increased by 25mm (1") - 50mm (2").

For greater flexibility, chairs can be built quickly to suit user requirements/ changes in the users condition.



Anti-tip bars

Width adjustable

Spare components can be stocked rather than completed chairs, offering greater flexibility to suit user requirements and changes in the users condition.

Choice of seat styles

Aperture - For toileting Horseshoe - For washing/toileting Flat padded - For shower use only

Swing away, removable height

and swing clear for easy carer access/hoisting.

Swing away and removable locking armrests


Easy side transfer allowing the user to move onto the shower chair without the need for manual handling.



Swing away armrests



Technical Information

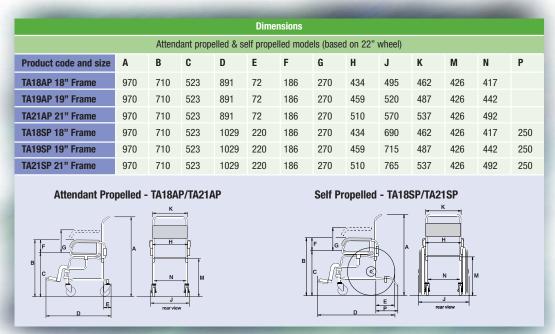


Note: Dimensions M can decrease by up to 15mm if the maximum load is applied.

A,B,C & M can be increased by 25mm or 50mm by the use of leg extension pins.

Dimension J (overall width) can be reduced by 30mm (on Self Propelling models only) if wheel rims are removed.

Note: There are three different wheel size options (20", 22" and 24") which give different seat heights and clearance dimensions. Please contact Chiltern Invadex for guidance.

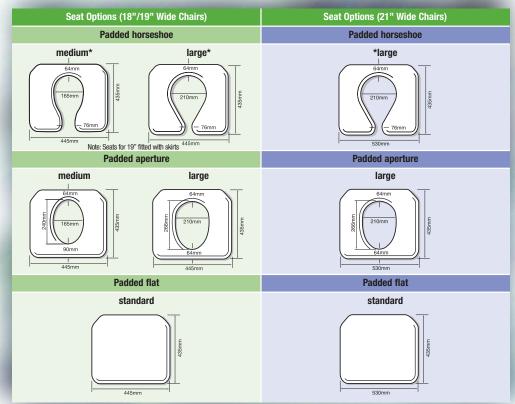

*Extended seat options available (50mm longer seat)

Safety: The TransAqua Self Propelled and Attendant Propelled shower chairs carry the CE mark and meet the essential requirements of the Medical Device Directive (93/42/EEC).

Dimensions shown are in MM for guidance only and are for unladen chairs.

Skirts available on all aperture and horseshoe seats

Maximum User Weight

Maximum user weight - 200 kgs (30 stones)

Total weight of chair

- 21kgs (Self Propelled) 16kgs (Attendant Propelled)
- No individual component weighs more than 6.5 kgs



Chiltern Invadex (UK) Limited

Chiltern House 126 Churchill Road Bicester, Oxfordshire OX26 4XD

Tel: 01869 365500 Fax: 01869 365588

E-mail: sales@chilterninvadex.co.uk Web: www.chilterninvadex.co.uk

Overseas Enquiries

Tel: +44 (0) 1869 365500 Fax: +44 (0) 1869 365588



Other products available from Chiltern Invadex include:

Showering and Bathing

Wet Floor Systems
Shower Cubicles
Shower Accessories
Shower Seats
Shower Trolleys and Stretchers
Easy Transfer Baths
Level, Easy and Step-in Showers

Moving and Handling

Ceiling Mounted Hoists Slings Mobile Hoists

Brochure Hotline: 01869 365500

Please contact your local distributor





Bicester is BS EN ISO 9001:2008 approved (FM 553189).