

How to use the Enteral UK adaptors




Product Code: TRN202

Converts ENFit to a Catheter tip syringe

Single Patient Use

The adaptor should be cleaned meticulously using hot soapy water before being rinsed thoroughly with clean water and subsequently allowed to “air-dry”. While no guarantee can be given, the adaptor is expected to last for a period of up to 2 weeks.

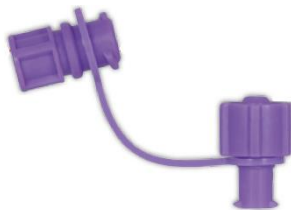

Product Code: ADFR2ISO

Converts a male reverse luer connection to ENFit



Product Code: ADISO2FR

Converts a male ENFit connection back to reverse luer



Product Code: UAD2ISO

Converts funnel end connection to ENFit

Single use

The adaptor is designed to be attached to the feeding tube and then left in situ. Once attached it should be left in place. The adaptor must be discarded if it becomes detached from the feeding tube.