

ONE YEAR LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2002 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China



Nimbo Pelvic Stabilizer

*Wenzel*ite*Re/hab*



Item # KA 1050

For use with all Nimbo Posterior Walkers

Instructions & Warranty

www.drivemedical.com

Installation:

1. Place pelvic stabilizer over rear frame of Posture Walker.
2. Line up holes (A) (Fig 1) of pelvic stabilizer with holes (B) (Fig 2) of Posture Walker.
3. Remove knobs (F) (Fig 3) from plastic bag, insert into holes (A) and tighten.

Adjustments:

1. Pelvic Stabilizer can be mounted either under (Fig 3) the frame or over (Fig 4) the frame for height adjustment.
2. Loosen knobs (E) for lateral width adjustment.
3. Adjust the belt loops to lengthen or shorten the belt.

Fig 1

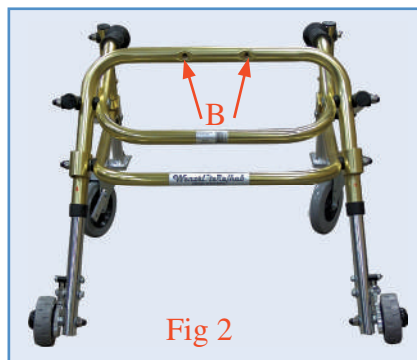
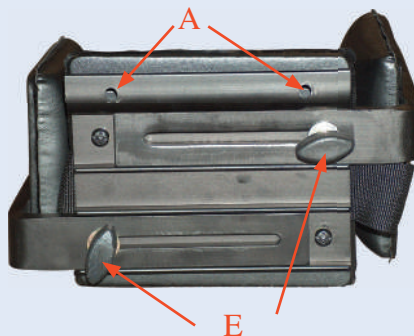


Fig 2

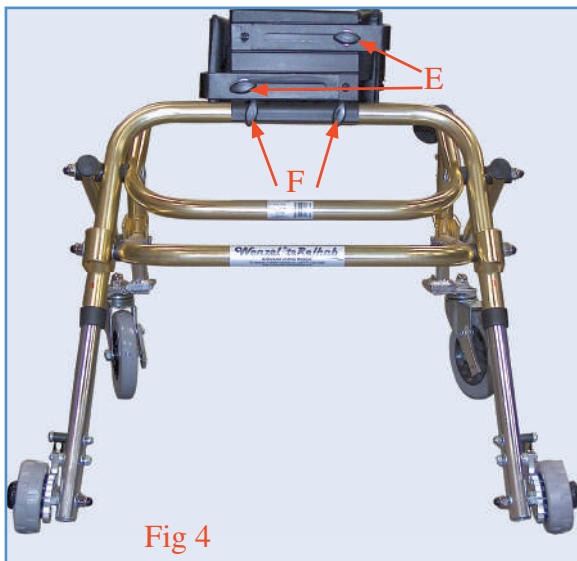


Fig 4

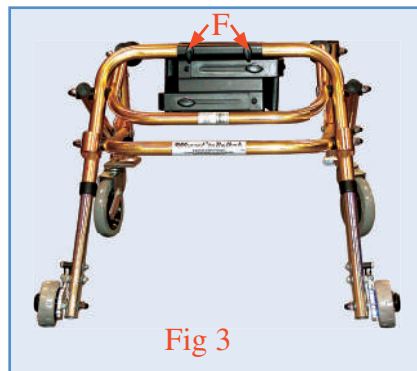


Fig 3