

## Manual LCD display Firefly 2014:

### **1. How do I turn on Firefly?**

On the battery there is a gray switch (with 0 and 1 position).  
Put the battery switch to position 1.

Next to the LCD display, is a keyboard with three keys.  
Press the middle button for 2 seconds.  
The Firefly is turned on and the LCD display will come to life.

### **2. What is indicated on the LCD display?**

The time is in the top right corner.

The horizontal column on the top left, indicates the battery level.  
Fully charged the column in its entirety black.

"Assist" level 1.

This means that the Firefly is in the lowest speed mode.  
At level 1, then reacts Firefly softly on the throttle.  
In the lowest speed mode, you can use the Firefly safely  
Inside the house or shop, with its maximum speed of 6.4 km/h.

In the center is a figure 00.0 Km / h, indicates the speed.

At the bottom ODO indicates the number of km the Firefly  
has driven in total.

### **3. How do I change the speed modes of the Firefly?**

The Firefly has five speeds.

When the top key is pressed on the keyboard, The value of  
speed at "Assist" jumps from 1 to 2. If you repeat it, the speed jumps  
from 2 to 3, and so on, until 5.

When the bottom key is pressed on the keyboard, the value at Assist  
jumps a value lower. For example from 5 to 4.

With regard to the different speeds:

The higher the gear, the engine responds more agile to the use of the throttle, and the the maximum speed of the Firefly is become higher.

At a value of 2, the maximum speed becomes 9.6 km / h

At 3, the maximum speed climbs to 13.3 km / h.

A value of 4, the maximum speed is 18.0 km / h.

A value of 5, the top speed is 25.0 km / h

#### **4. How to turn on the headlamp?**

You press the upper and the middle key at the same time for 2 seconds or a little longer.

Not only the lamp, but also the display lights up.

By doing the same thing again and again, the headlight and the display are turned off.

#### **5. How do I see the distance I covered?**

You press a short time on the middle button, and the ODO changes to TRIP. The Trip value indicates the number of kilometers you have traveled that day.

#### **6. How do I adjust the time?**

When the Firefly is turned on, you press on the upper and lower key simultaneous for 2 seconds. The indication of the hours starts to blink.

Press the top button, to move the hours one up.

Press the bottom button, to move the hour one down.

Press the middle button, then you move to the minutes, who start to flash.

Then press the top button, then moves to one minute up.

Then press the bottom button, it moves one minute down.

Then press (longer time) on the middle button, and the display returns to the operation mode.

**Warning:** Look out when you press the bottom button.

When you press the bottom button more than 2 seconds, then see a running man on the display.

At the same time, the Firefly starts moving forward.

This mode is designed for cyclists, who have to push their bicycle up a hill, so the motor helps them with making the climb, while walking next to their bicycle.

I don't see the wheelchair user benefit from this modus.

So for the Firefly this is not a useful function.